LANAFORM®

Safety Handle

Dual Grip Handle

Maximum
tension: 80 kg

Traction
maximale :80 kg

Maximum-
spanning: 80 kg

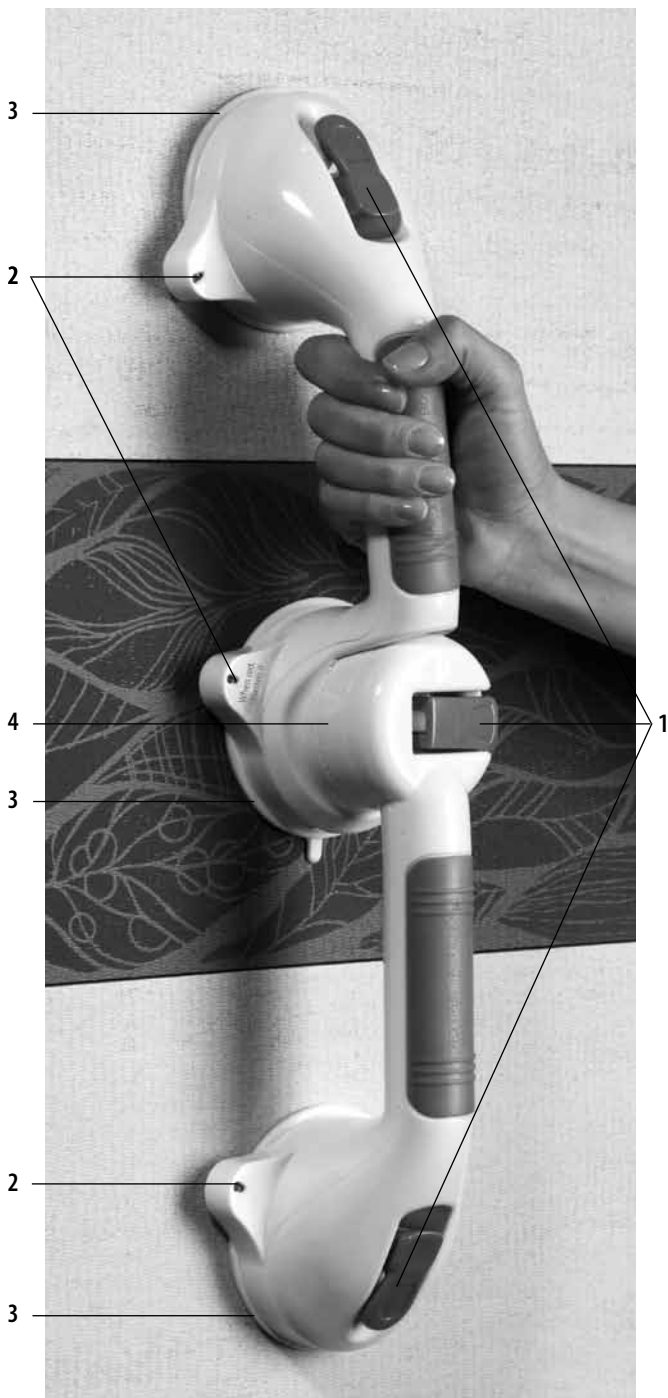
Figure 1

Figure 2

Figure 3

Figure 4

Figure 5

Figure 6

Figure 7

SAFETY HANDLE

Instruction manual

We would like to thank you for purchasing the LANAFORM SAFETY HANDLE.

This jointed handle with double safety will guarantee you all of the support and stability you need in your washroom or anywhere else.

This handle will help you move about and get up safely anywhere the daily risks of slipping or falling are great. Thanks to its 3 mechanical suction devices, it instantly attaches itself to glass, fibreglass or porcelain, with no need for puncturing or screwing.

Special feature: colour indicator light to ensure proper fastening.

PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE USING YOUR SAFETY HANDLE FOR THE FIRST TIME, ESPECIALLY THIS FUNDAMENTAL SAFETY CODE:

- Always place the SAFETY HANDLE on a surface that is perfectly flat and non-porous: use on tile, glass, fibreglass or porcelain. It cannot be placed on any other type of surface.
- Do not attach the SAFETY HANDLE to movable surfaces or mortar or tile joints.
- Before each use of the SAFETY HANDLE, check the wear and functioning of the suction discs. Do not use the SAFETY HANDLE if you have doubts about its integrity.
- Before each use of the SAFETY HANDLE, observe the suction force safety indicators. They must always be GREEN when the suction force levers are engaged. Depending on the type of surface, a properly tightened handle can slowly become loose. This is why it is safer to check the suction force safety indicators and to ensure proper adhesion of the handle on the chosen surface.
- This handle can only be used for support purposes. It cannot bear your entire weight.
- The maximum resistance of the SAFETY HANDLE is 80kg for vertical exertion.
- The SAFETY HANDLE must be cleaned regularly. To do this, read the cleaning instructions in this manual.
- Do not expose the SAFETY HANDLE to prolonged sunlight.
- Do not use sharp or pointed objects to release the handle.
- Leave the levers in the off position when the handle is not in use.
- Do not disassemble the device and do not attempt to repair it yourself.
- Should you encounter problems with your unit, please contact your sales representative.
- This unit is not intended to be used by people, including children, whose physical, sensorial or mental skills are impaired, or by people who do not have the experience or knowledge to use it safely, unless they are supervised or have been given prior instructions by an intermediary responsible for their safety about how to use this unit. Children should be supervised at all times to make sure that they do not play with the unit.

- Use the SAFETY HANDLE exclusively for its purposes, as described in this manual.

DESCRIPTION OF THE DEVICE (see diagram on page 2)

1. Suction force levers
2. Safety signals (suction force)
3. Suction discs (suction devices)
4. Rotation of 2 handles

INSTRUCTIONS FOR USE

Instructions for use:

1. Clean the chosen flat surface with a cleaning product and let it dry.
2. Check the proper functioning of the device. Turn the three suction force levers to the on position (figure 1). Then position the SAFETY HANDLE in the desired location. Hook each suction disc separately (figure 2) while being sure to properly compress the disc before pushing the lever down.
3. Do not place the handle on mortar or tile joints (figure 3).
4. After setting the handle, the safety indicators must be green (figure 4). The suction force should then be noticeably strong; please check to make sure it is (figure 5).

Principles:

- When the safety indicators are GREEN, the handle has been attached properly.
- When the safety indicators are RED, the handle has not been attached properly and must be refastened.

How to remove the SAFETY HANDLE:

1. Turn the three suction force levers to the off position (figure 6).
2. There may be some suction force present. If this is the case, lift a suction disc by the strip with your fingers (figure 7).
3. Leave the levers in the off position until the next time you use the SAFETY HANDLE.

MAINTENANCE AND STORAGE:

Clean the device with a soft and damp cloth. Use cleaning devices that are non-abrasive in order to prevent damaging the device.

Store this device in a dry place and make sure that the three levers are in the off position.

TECHNICAL INFORMATION:

Length (completely open): 49 cm

Height: 8 cm

Diameter of suction device: 9.5 cm

LIMITED WARRANTY:

LANAFORM® guarantees that this product will not suffer from any material or manufacturing defect for a period of two years after purchase, with the exception of the following cases:

This warranty for a LANAFORM® product does not cover damages that result from incorrect or excessive usage, an accident, the addition of an unauthorised accessory, a modification to the product or any other situation beyond LANAFORM®'s control.

LANAFORM® denies all responsibility in cases of accidental, successive, or special damage.

Upon reception, LANAFORM® will repair or replace, as the case may be, your device and return it to you, postage paid. This guarantee is only available through the LANAFORM® Service Centre. If maintenance is performed on this product by a service other than the LANAFORM® Service Centre, the warranty is null and void.

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