LombaStab®

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EASY-TO-ADJUST REINFORCED LUMBAR STABILISATION BELT

Characteristics of the LombaStab® belt:

For greater efficacy:

- A reinforced back panel (1) and strapped elastic fabric (2) providing increased lumbar stabilisation.

For greater practicality:

- A sliding handle system (3) making it easier to adjust the level of compression.

For greater comfort:

- An anatomical cut making the belt more ergonomic.
- An inner back panel lined with Coolmax® for better comfort.

Indications:

- Non specific low back pain treatment (Acute/ Subacute).
- Adjuvant treatment for secondary low back pain.

Contraindication:

- Do not apply the product in direct contact with broken skin.

Fitting instructions:

The belt should be worn on top of a t-shirt or a thin vest.

- Open the belt and lay it out completely flat; you must be able to read the Quick Lacing system (4) and LombaStab® (4) logos the right way round.
- Check that the two tightening handles (3) are fully retracted; detach them from the velcro on the abdominal panel if necessary and pull on both ends of the belt.
- Once you have prepared it, place the belt around your waist, on top of a garment such as close-fitting vest (a):
 - The Quick Lacing system and LombaStab® logos must be located on the outside surface of the belt at the back (i.e. not against the skin).
 - The lacing section must be centred over your spine: the pads should be positioned on either side of the spine.
 - The bottom of the belt must stop at the top of the crease between the buttocks.
- Fasten the belt using the wide velcro straps, right side over left side, without pulling too tight or applying any particular pressure (b - c).
- Adjust the tightness using the two handles: pull them forwards then fold them flat against the front of the belt, where they attach like velcro (d - e).
- You can use these two handles to adjust the level of compression as required throughout the day.

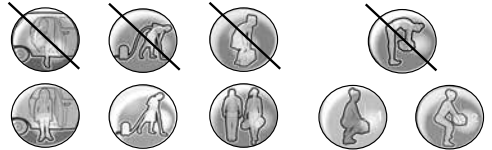
Precautions:

- Follow the advice of the Health Care Professional who prescribed or supplied the product.
- It may be necessary to shape the stays; this must be done by the health professional.
- Consult your health professional if discomfort occurs.
- Store at room temperature, preferably in its original packaging.

- For hygiene and performance reasons, do not re-use the product for another patient.

Habits to get into

Here are a few simple movements to help you educate and protect your back.

Washing instructions:

- See product label.
- Machine wash at 30°C, delicates cycle, preferably in a wash bag.
- Fasten Velcro tabs before washing.
- Do not use detergents, fabric softeners or aggressive products (products containing chlorine).
- Squeeze out excess water.
- Dry away from heat sources.

Composition

- See product label.

Keep out of the reach of children.

The presence of resistant laces means that the belt must not be worn close to machines with uncovered moving parts.

Additional information

For more detailed information about the LombaStab® belt or low back pain, please consult the Thuasne www.thuasne.fr website

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**THUASNE SAS – 118 rue Marius AUFAN – CS 10032