# THUASNE

---

## **LOMBAX® ACTIVITY**

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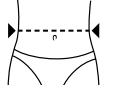
Conserver cette notice.

# LUMBAR BELT FOR INTERMITTENT ACTIVITY

ENG

## Properties :

- Removable straps for periods when support is temporarily not required ; it is possible to keep belt on due to its suspenders.
- Stiff conformable posterior stays. (2)
- Optimized back comfort with a massaging effect.
- Additional straps for adjusting the level of back support. (3)
- Hand loop to make it easier to put the product on.

## Indications :

- Initial compression for lumbar pain and sciatica
- Prevention of lumbar pain (exertion, poor posture), particularly in active people who experience intermittent heavy loading of the spine
- Lumbar compression following surgery.

## Fitting instructions :

- When fitting for the first time :
  - Unfasten the lateral Velcro straps.
  - Put on the belt, with the inside label upwards.
  - Using the hand loop, fasten the ventral Velcro strap, adjusting it to the desired level of support.
  - Adjust dorsal support by fastening the additional lateral straps at the front using the finger loops.
  - Position and adjust the braces.

## • Thereafter :

- With the various adjustments now having been correctly made, put on and remove the belt simply using the ventral strap.
- When support is temporarily not required, it is possible to keep belt on due to its suspenders. This helps to minimize the hassle of constantly putting it on and taking it off.

## Precautions :

Follow the advice of the professional who prescribed or supplied the product. It may be necessary to reshape the stays. Consult your professional if discomfort occurs.

Store at room temperature, preferably in its original packaging.

Re-use of this device for the treatment of another patient is strictly not recommended due to hygiene- and performance-related problems. If a decision is taken to re-use this device, then the person taking the decision is solely responsible and the Thuasne warranty will automatically become void.

## Washing instructions :

- Wash only with soapy water, without detergent or bleach.
- Squeeze out excess water.
- Dry away from heat sources.

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