


Instructions de montage:

Étape 1: Cette barrière de lit peut être placée des deux côtés du lit. Dans cette figure, la barrière de lit est montée sur le côté gauche du lit.



Étape 2: Sortir toutes les pièces de la boîte et les placer sur une surface plate.

Étape 3: Vérifiez attentivement que toutes les pièces sont incluses dans le carton.

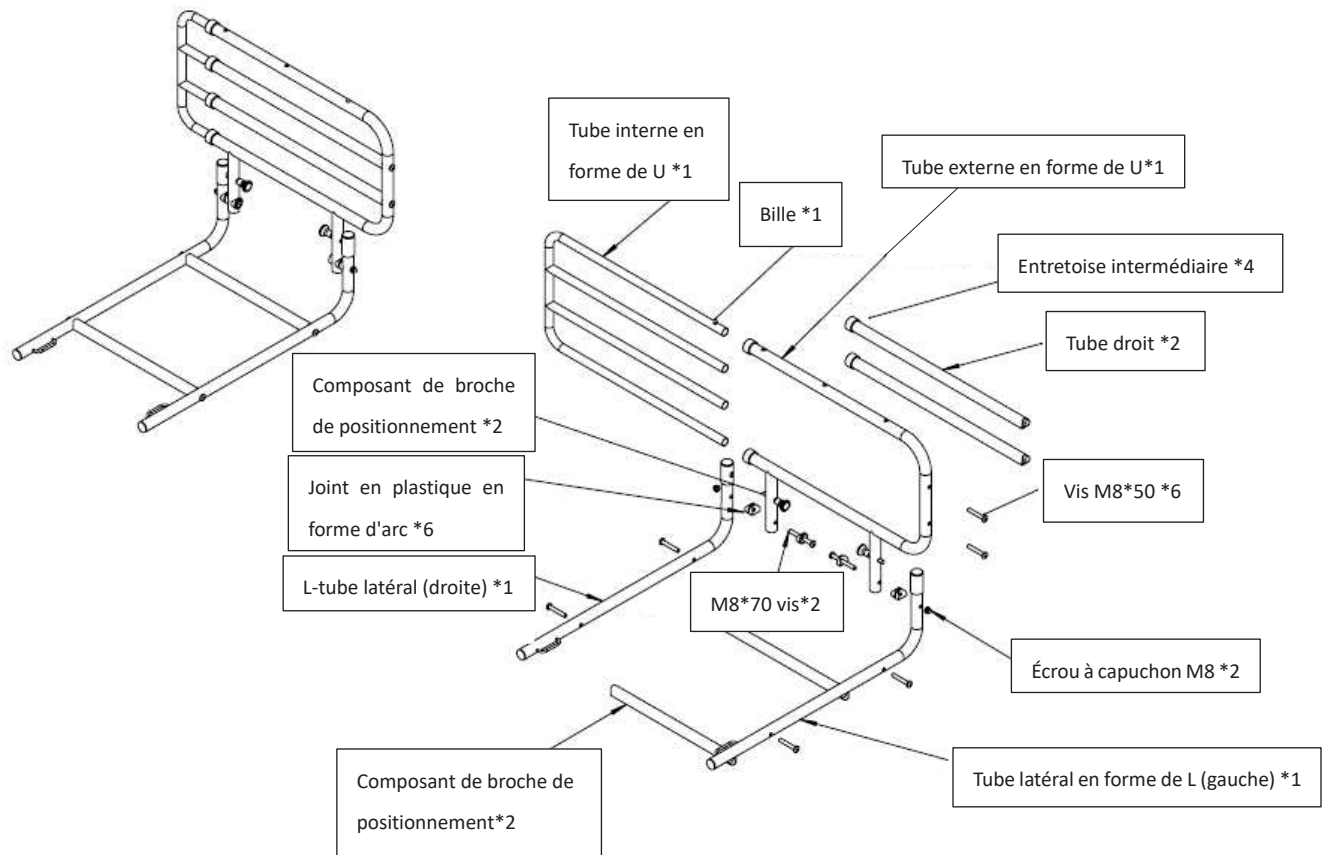

Étape 4: Fixer les tubes en L à la structure principale à l'aide des vis, des joints en plastique et des écrous

Étape 5: Après cela, placez la ceinture de sécurité dans les tubes en forme de L, comme indiqué sur l'image.

Étape 6: Finalement fixer la barre transversale avec les vis et les boulons à chaque barre.

Caractéristiques techniques

Matériau : Acier
 Longueur réglable : 67cm à 107cm
 Hauteur : 58cm



INSTRUCTIONS MANUAL EN

Warning:

1. Maximum user weight: **150kg weight limit**
Do not exceed the weight limit stated as doing so could put the user at risk.
2. Please examine fully for any damage or faults and read the manual carefully before using the bed rail. If you suspect a fault or damage, do not use, and contact your supplier immediately.
3. Do not use it without any mattress or the thickness of mattress less than **10CM**.
4. Clean regularly with soft cloth and do not use abrasive cleaners.


Assemble instructions:

Step 1: This bedrail can be placed on both sides of the bed. In this figure, the bed rail is showed on the left side of the bed.



Step 2: Remove all parts from the box and place them on a flat surface.

Step 3: Carefully check that all parts are included in the carton.

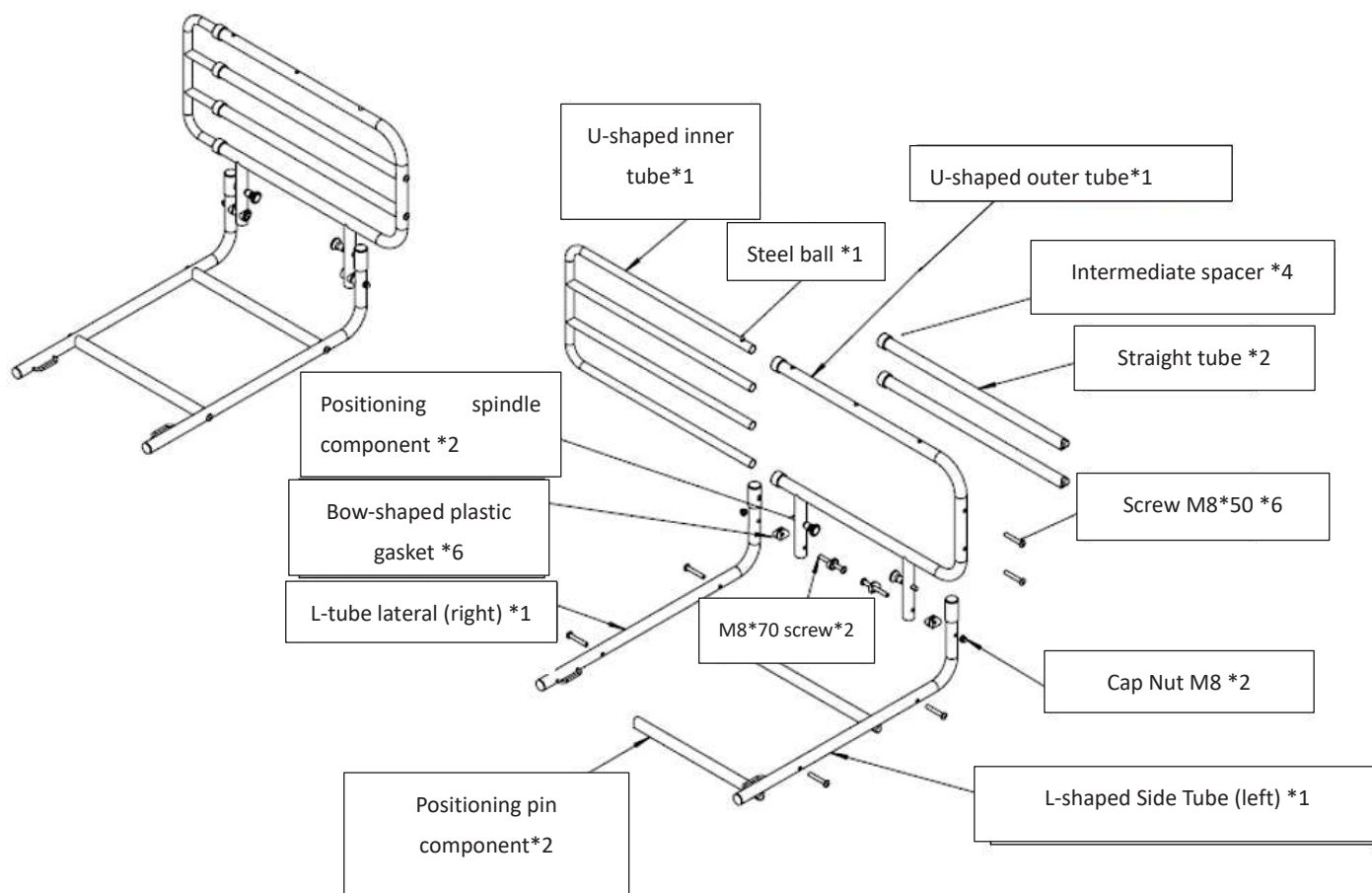

Step 4: Fasten the L-shaped tubes to the main structure using screws, plastic gaskets, and nuts.

Step 5: After this, place the seat belt in the L-shaped tubes, as shown in the picture.

Step 6: Finally fix the crossbar with screws and bolts to each bar.

Technical Specifications

Material	:	Steel
Adjustable length	:	67cm to 107cm
Height	:	58cm

Distribué par/Distributed by :
 IDENTITES
 Boulevard de la Chanterie – ZA Pôle 49
 49124 Saint Barthelemy d'Anjou
 Téléphone : 02.41.96.18.48