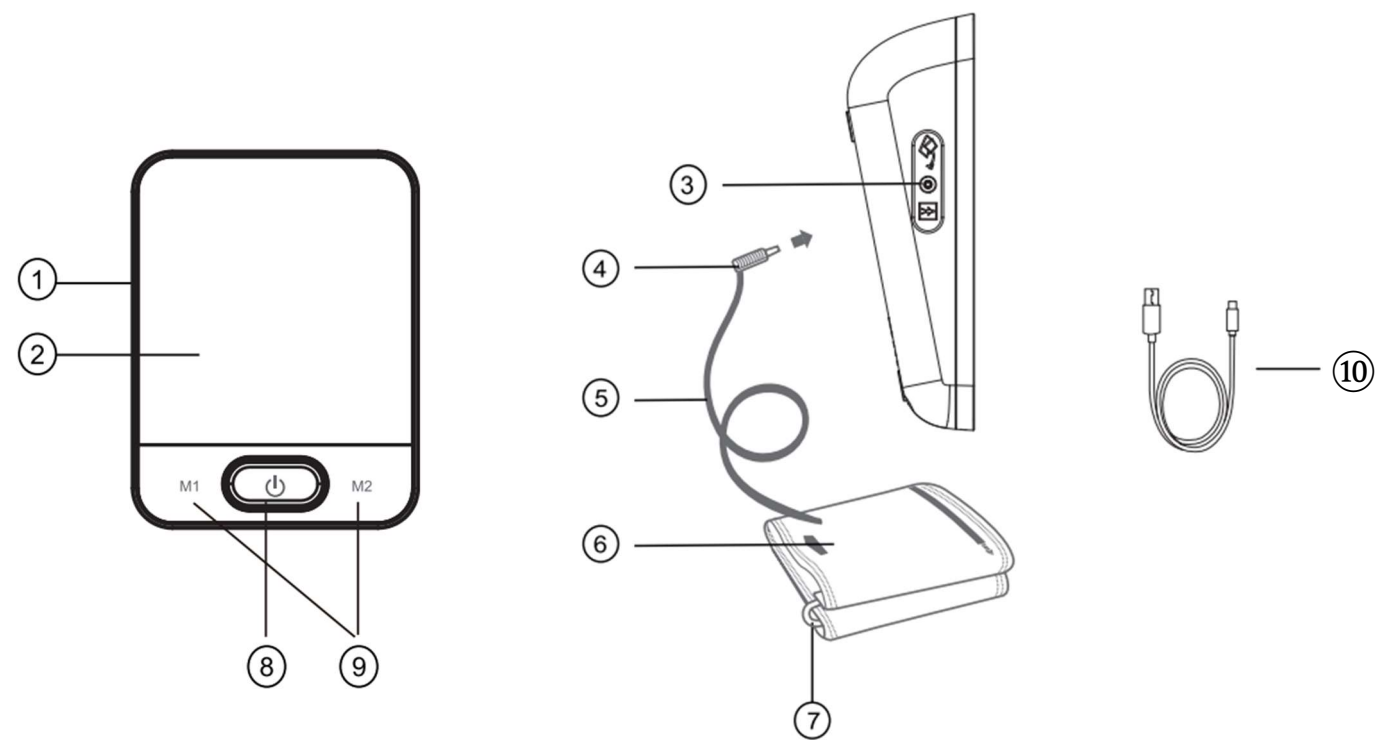
	FR	DE	NL	EN
①	Tensiomètre	Blutdruckmessgerät	Bloeddrukmeter	Pressure monitor
②	Ecran	Bildschirm	Scherm	Screen
③	Branchement du tuyau d'air	Anschluss Luftschlauch	Luchtlandgaansluiting	Air tube hole
④	Connecteur de tuyau	Anschlussstelle	Slangaansluiting	Air plug
⑤	Tuyau d'air	Luftschlauch	Luchtslang	Air tube
⑥	Brassard	Manschette	Armband	Arm cuff
⑦	Anneau en D	D-Ring	D-ring	D-ring
⑧	ON/OFF	Ein/Aus	Aan/uit-schakelaar	ON/OFF
⑨	Mémoire M1/M2	Tasten "M1/M2"	Proef "M1/M2"	M1/M2 Memory
⑩	Câble USB-C	Type-C Kabel	Type-C kabel	USB-C cable

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TALKING BLOOD PRESSURE MONITOR

EN

Warning: Please read this instruction manual carefully before using the instrument and keep it for future use.

A. SAFETY INSTRUCTIONS

The device is intended for domestic use only.

If you have any medical concerns, talk to your doctor before using the blood pressure monitor.

Use the device only as directed in the package leaflet.

If you suffer from any medical conditions, for example arterial occlusive disease, please consult your doctor before use.

The device should not be used to control the frequency of a pacemaker.

Pregnant women should take the necessary precautions and take into account their personal limitations. Talk to your doctor if necessary.

If you experience any discomfort during the measurement, such as arm pain, deflate the cuff immediately. Undo the cuff and remove it from the arm.

This appliance is not intended for persons (including children) with reduced physical, sensory or mental capabilities or lack experience and/or knowledge, unless they are supervised by a person responsible for their safety or have been instructed in the use of the appliance.

Children should be supervised by making sure they are not playing with the device.

The blood pressure monitor should not be used near devices that emit significant electrical radiation, such as radio transmitters, mobile phones, or microwave ovens.

If there are any problems, do not repair the device yourself, as this will void any warranty rights. Contact authorized centers to carry out the repairs.

Store the device away from moisture. If moisture gets into the appliance, remove the batteries immediately and stop using the appliance. In this case, contact your dealer or inform us directly.

If you are not using the device for a long time, remove the batteries.

B. HOW TO USE IT

1) Insert or replace batteries

- Open the battery cover
- Insert 4 x AA LR6 1.5V batteries. Please respect the polarity of the batteries.
- Replace the battery compartment cover.

As an alternative to batteries, you can use a power adapter as a power source. The power adapter is optional for this unit.

- Plug the adapter cable into the input on the right side of the monitor.
- Plug the adapter plug into the power outlet.
- To remove the adapter, first unplug the adapter plug from the power outlet, and then unplug the cable from the monitor input.

2) To set date and time

This function provides the exact measurement time for each measurement. To get an accurate date and time, the user must set the date and time correctly before using this device for the first time.

The operating procedure for the date/time preset is as follows:

1. When the device is in standby mode, press and hold the "M1" button, then press "⏻" at the same time and the year will flash as shown in Fig.1.
2. Press "M1" to increase the number and "M2" to decrease the number, then press "⏻" to confirm.
3. When the year setting is finished, the month number will flash automatically. Please follow the instructions above to set the month, date, and time as shown in Fig.2.
4. If you want to change the date and time, repeat steps 2 and 3.

NOTE: If the unit is in date/time setup mode and there is no operation, the unit will automatically save the operation and return to the power off mode

3) Setting the voice function

The device has a voice function. To enable or disable the voice feature, please follow the steps below:

1. When the device is in standby mode, press and hold "M1", then press "" twice at the same time ⏻ until "ON" or "OFF" is displayed on the LED screen and "🔊" flashes as shown in Fig. 3;
2. Press "M1" to turn the voice function on or off;
3. Press "⏻" to confirm.

NOTE:

1. When the device is used for the first time, the voice function is enabled by default.
2. In the "VOICE FUNCTION SETTING" mode, the device will automatically save the setting and switch to shutdown mode after 30 seconds if no other operation is performed.

4) Setting the language

There are five languages to choose from for voice announcements.

To set languages:

1. When the device is in standby mode, press and hold "M1", then press "" three times at the same time ⏻ until the LED voice icon appears, as shown in Fig. 4;
2. The device will display "L1" (French is the default language when first using the device) with the voice prompt. Press "M1" to select another language;
3. Press "⏻" when you are done setting it. If you want to change the language, please repeat the steps above.

L1-French L2-English L3-Spanish L4-Russian L5-German

NOTE:

Fig.1

Fig.2

Fig.3

Fig.4

T 5870 / LD-587

1. The "L1" LED icon indicates that the language is French. There will be a demonstration voice "We wish you good health" in French. Other "L2-L5" icons use a demo voice in the corresponding languages.
2. Voice function increases power consumption and shortens battery life.
3. In "LANGUAGE SETTING" mode, the device will automatically save the setting and enter off mode after 30 seconds if no further operation is performed.

5) To apply the arm cuff

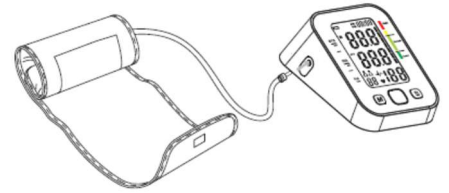
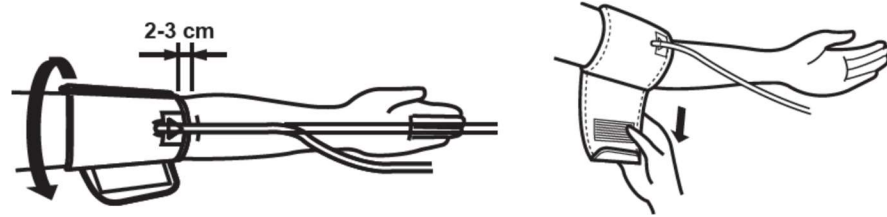
Connect the air hole to the monitor.

Pull the end of the cuff through the loop outwards (away from your body), then "scratch" the band around your arm.

Place the cuff around your bare arm 2-3cm above the elbow joint.

While sitting, place the palm of your hand upside down toward the ceiling while extending your arm on a flat surface such as a desk or table. Place the hose in the axis of your arm, aligning it with your middle finger.

The cuff should fit easily and comfortably on your arm. You should be able to insert a finger between your arm and the cuff.

Once the cuff is properly installed, plug it into the blood pressure monitor's main unit.

The cuff air hose should be plugged into the left side of the blood pressure monitor when the screen is facing you. The connection to the right of the blood pressure monitor is designed to fit an AC/DC adapter.

Body posture during measurement

Sit comfortably for measurement

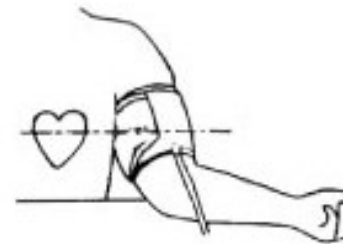
has. Sit with your feet flat on the floor, do not cross your legs.

b. Place your palm upside down toward the ceiling, extending your arm over a flat surface such as a desk or table.

c. The middle of the cuff should be at the level of the heart

Once done, press the Start button. The device will automatically squeeze your arm, **keep calm relax. Don't talk.**

Please wait until the screen shows the final results.

Wait at least 5 minutes between each measurement, otherwise you will get the wrong data.

has. Sit in a comfortable position with your legs uncrossed, the soles of your feet flat on the floor and your back resting on the back of your chair. At the same time, your elbow should be on a flat surface and the wrist cuff should be at the level of your heart

b. Release your wrist and turn your palm face up.

c. Relax, stand still and do not speak while taking the measurement, as this may distort the results.

NOTE:

- This appliance is intended for use by an adult. Do not allow children to use this appliance.
- For reliable and reference blood pressure monitoring, it is recommended to take the measurements daily at the same time.
- In order to obtain a resting blood pressure measurement, do not eat, drink alcoholic or caffeinated beverages, smoke, exercise, or take a bath at least 30 minutes before taking the measurement, knowing that your blood pressure depends on these parameters. Stress also increases tension.
- It is recommended that you do not get physically tired when taking the measurement.
- Take measurements in a quiet place at room temperature.
- Remain still and without speaking when taking measurements.
- Always wait at least 5 minutes between each use to allow blood circulation in the wrist to return to normal. Depending on your physiological conditions, you may have a longer waiting time.
- If the blood pressure monitor detects an abnormal condition, it stops the measurement and displays an error code. See "Troubleshooting" for more details.

• This device bases its measurements on your heartbeat. If you have weak or irregular heartbeats, the blood pressure monitor will likely have difficulty determining your blood pressure. (Irregular beats are defined as beats that vary by 25% from the average of all beats during the measurement).

6) To measure your blood pressure

1. Plug the air hole into the air tube hole. Take 3-5 deep breaths and relax before the measurement. Do not speak or move your arm. When the device is turned off, press "⏻" to enter sleep mode. The LED displays the time and date alternately.
2. When the device is in standby mode, press "⏻" and all icons will appear on the screen as shown in Fig.5. Then "0 mmHg" appears on the screen. The pump will then begin to inflate and the display will display the pressure value.
3. If the arm cuff is too loose, it may result in unreliable measurement results or the measurement may not be started. The arm cuff check indicator (⊗) flashes to determine if the cuff is tight enough. The symbol "⊗" appears when the cuff is correctly applied during the measurement, as shown in Fig.6. Otherwise, a machine error will appear if the cuff is too loose. For more information, see Error Information.
4. The pressure in the arm cuff then increases up to the working pressure. Then the pump stops and the pressure is quickly reduced.

NOTE: The device will automatically inflate to a higher pressure if the inflation pressure is not sufficient to determine the measurement results.

5. The heart symbol "♥" starts flashing with each heartbeat as soon as the pulse is detected, as shown in Fig.7.

6. After the measurement, flash "1" or "2" to remind the user to record the values of systolic pressure, diastolic pressure, and pulse rate as shown in Fig.8. Press "M1" or "M2" to save the readings to the appropriate storage space.

7. If an irregular heartbeat is detected during the measurement, the LED display will display the "⚡" icon to alert the user of a cardiac arrhythmia, as shown in Fig.9.

8. "Motion detection" helps the user stay calm and alerts you to any body movement during the measurement. The symbol "🚶" appears when "body movement" has been detected during and after such a measurement.

NOTE: It is strongly recommended that you repeat the measurement when this icon appears.

10. 9. Press "⏻" to turn off the device. Please rest for at least 3 minutes before taking another measurement.

Fig.5

Fig.6

Fig.7

Fig.8

Fig.9

WHAT ARE SYSTOLIC PRESSURE AND DIASTOLIC PRESSURE?

When ventricles contract and pump blood out of the heart, blood pressure reaches its maximum value, the highest pressure in the cycle is known as systolic pressure. When the heart relaxes between heartbeats, the lowest blood pressure is diastolic pressure.

7) Memory recovery

1. The pressure monitor can store 100 readings in both "M1" and "M2", and automatically calculates the average value of the last 3 measurements. When memory is full (100 reads stored), the oldest metric is automatically replaced with the newest. The device's memory is not erased even if the power is turned off;
2. After a measurement or when the device is in standby mode, the user can press "M1" or "M2" to retrieve a recorded reading. Press "M1" and the display will show the average value of the last 3 measurements, as shown in Fig.10;

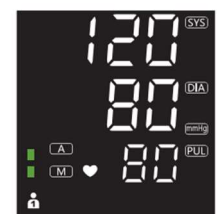
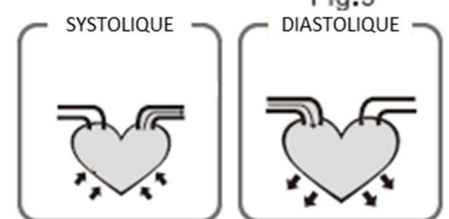
Fig. 10

3. Press the button again, the screen will show "01", which means the last measurement and time of the measurement, then switch to another screen to show the readings as shown in Fig.11;

4. Press "M1" again, the screen will show "02", which means the penultimate measurement.

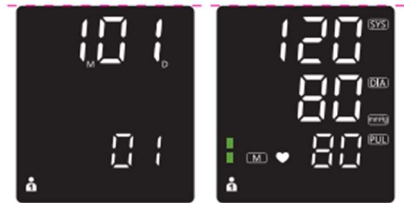
Fig. 11

Fig. 12

Fig. 13

ERASE MEMORY

When the device is turned off or in sleep mode, press and hold the "M1" button for at least 5 seconds. The display displays "CL", which means that all stored measured values have been deleted, as shown in Fig.12.

In memory mode, press and hold "M1" or "M2" for at least 5 seconds. The display shows "CL", which means that all measured values recorded in this group have been removed, as shown in Fig.13.

C. ERROR AND LOW BATTERY INFORMATION

INDICATIONS	PROBLEMS	SOLUTIONS
 	ERO0: The device doesn't find the zero point.	Contact the seller or manufacturer.
 	ER01: Blood pressure and pulse rate cannot be measured. ER02: The armband is not tight enough. ER03: Body movement was detected.	Press "⏻" to return to sleep mode. Make sure the cuff is properly put in place and the hose plug is firmly inserted. Repeat the measurement according to the recommendations in the instructions for use.
	The batteries are low, but the device can still operate normally.	Be prepared to change the battery.
	The batteries are low.	Replace all batteries.

TROUBLESHOOTING

SYMPTOM	CHECKPOINT	MEDICINE
No display when turned on.	The batteries are empty.	Replace all batteries.
Inflation stops, then starts again.	Automatic inflation to ensure correct measurement. Did you speak or move your arm (or hand) during the measurement?	Remain still and calm during the measurement.
The reading is extremely low or high.	Is the cuff at the same height as the heart? Is the cuff correctly put on? Did you extend your arm during the measurement? Did you speak or move your arm (or hand) during the measurement?	Make sure your posture is correct. Roll up the cuff properly. Relax during the measurement. Remain still and calm during the measurement.
The pulse is too low or too high.	Did you speak or move your arm (or hand) during the measurement? Did you take the measurement directly after training?	Remain still and calm during the measurement. Repeat the measurement after a rest period of at least 5 minutes.

D. DECLARATION

Compliance information for each EMC test

Electromagnetic emission (home, health, environment)	
Emission test (IEC 60601-1-2:2014): Compliance	Compliance

Conducted and radiated RF emissions	CISPR 11 Group 1 Class B
Harmonic emissions IEC 61000-3-2	Class A
Voltage fluctuations/flicker emissions IEC 61000-3-3	Conformable

Compliance information for each EMC test

Declaration of electromagnetic immunity (home health environment):		
Immunity Test	IEC 60601 Test Level	Level of compliance
RF Driving IEC 61000-4-6	3V 150 kHz to 80 MHz 6 V in the ISM and amateur radio bands between 0.15 MHz and 80 MHz	3V 150 kHz to 80 MHz 6 V in the ISM and amateur radio bands between 0.15 MHz and 80 MHz
HF radiation IEC 61000-4-3	10 V/m, The band from 80 MHz to 2.7 GHz also meets the requirements of Table 9 of the 60601-1-2:2014 standard	10 V/m, The band from 80 MHz to 2.7 GHz also meets the requirements of Table 9 of the 60601-1-2:2014 standard
Electrostatic Discharge (ESD) IEC 61000-4-2	±8 kV connector ±2 kV ±4 kV ±8 kV ±15 kV Luft	±8 kV connector ±2 kV ±4 kV ±8 kV ±15 kV Luft
Fast transients/bursts of power IEC 61000-4-4	±2 kV for power lines	±2 kV for power lines
Overvoltage IEC 61000-4-5	±0.5 kV, ±1 kV line-to-line	±0.5 kV, ±1 kV line-to-line
Voltage drops, short interruptions, and voltage fluctuations on power supply input lines IEC 61000-4-11	0% UT 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°; 0% UT 1 cycle and 70% UT 25 cycles single-phase: at 0°; 0% UT 250 cycles	0% UT 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°; 0% UT 1 cycle and 70% UT 25 cycles single-phase: at 0°; 0% UT 250 cycles
Magnetic field mains frequency (50/60Hz) IEC 61000-4-8	30 A/month	30 A/month

NOTE: The EUT is the mains voltage before the test level is applied. The following phenomenon continues to meet basic safety and essential performance requirements.

*UT: 230V ~/50Hz. The AEM pressure deviates from normal, but the value is still above 10 psi when the flow rate is 4.5 L/min.

**UT: 230V~ /50Hz. The EUT stops working when 0% UT is added, but can automatically return to normal mode.

- Using this appliance in close proximity or stacking with other equipment should be avoided as it may cause malfunction. In the event that such use is required, this and other appliances should be observed to ensure that they are functioning normally.
- Portable RF communication devices (including peripherals such as antenna cables and external antennas) should not be used within 30 cm (12 inches) of any part of this device, including cables specified by the manufacturer. Otherwise, the performance of that device may be affected.
- Under the specified immunity test conditions, the product can provide basic safety and essential performance.
- If critical performance is lost or compromised, additional measures are required, such as redirecting or moving the device.

E. CARE AND MAINTENANCE

Clean the product only with a dry or slightly damp, lint-free cloth.

Do not use abrasive cleaning products to clean the appliance.

It is necessary to protect this device from high humidity, direct sunlight, shocks, solvents, alcohol, and gasoline.

Remove the batteries if the device is stored for a long time and keep the batteries out of the reach of children.

This appliance is not washable. Never immerse the device in water or rinse it under the tap.

The cuff is delicate and must be handled with care. You can clean the cuff with a damp cloth for daily maintenance.

To prevent cross-infection when splitting the cuff, you can sterilize the fabric covering of the cuff with a sponge soaked in a 3% hydrogen dioxide solution. After prolonged use, partial discoloration of the surface of the cuff fabric occurs. Do not wash or iron the cuff.

WARNING: Under no circumstances should you wash the inner airbag!

7. Since neither the device nor the batteries are household waste, follow local recycling rules and dispose of them at a suitable collection point.
8. Do not open the unit as this may damage sensitive electrical components such as a complicated air unit. If you are unable to resolve the issue with the troubleshooting instructions, contact your dealer.

F. TECHNICAL

- Power supply: Type-C adapter DC 5V 1A, 4 x AAA battery
- Measuring range:
 - 40 to 180 mmHg (diastolic pressure DIA)
 - 60 to 260 mmHg (SYS systolic pressure),
 - 40 to 160 beats/minute (pulse rate PUL)
- Precision:
 - ±3 mmHg for static pressure
 - ± 5% pulse rate
- Operating Temperature/ Humidity: +10°C to +40°C, 85% and below, 800hPa to 1060hPa
- Storage Temperature/ Humidity: -20°C to +50°C, 85% and below, 500hPa to 1060hPa
- Memory: 2 users with 100 memory slots each

The European directive 2012/19/EC about electronic and electric waste, requires that you can't throw away defective domestic appliance with common waste. Used device has to be collect separately to optimize the recuperation rate and the recycling of the materials to reduce the impact on the health and the environment.

Precision and precautions about the battery use

- Throwing away a used battery in the environment or with household waste pollutes and avoids the recuperation of reusable materials.
- It is important to limit the battery consumption and to respect the following rules:
 - Favor alkaline battery (last longer than saline battery) and when it's possible use rechargeable battery.
 - Leave used batteries and accumulators in adapted containers that you can find in some retailers. In that way, metals don't pollute the environment and the health.
- Battery has to be inserted by respecting the polarity show on the apparel and the battery. An incorrect positioning can damage the apparel, cause leaking and even provoke fire and the explosion of the battery.
- To assure a good functioning, battery has to be in a good condition. If there is a problem, put new battery.
- Never try to recharge battery not made for this. It could leak, warm up, cause fire or explode.
- Replace all the battery at the same time. Never mix up alkaline battery with saline battery.
- Used battery has to be removed.
- Also, remove the battery if you're not using the apparel for a long time.
- Never try to short-circuit the terminal of a battery.
- Never throw away the battery in fire because they can explode.
- Take out the accumulator from the apparel before reload it.
- We recommend that an adult watch if a child change the battery to respect those rules.
- If you swallow a battery, go immediately to your doctor or the hospital.
- For the best ongoing performance, we recommend using only fresh, high-quality, alkaline batteries.

Declaration of Conformity

Captelec declares that this product T 5870 conforms to the guidelines 2014/53/EU, 2011/65/EU, 2014/35/EU and 2014/30/EU
The full text of the EU declaration of conformity is attached at the end of the package leaflet.
This declaration is invalid if changes have been made to the product for which we have not agreed.

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