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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

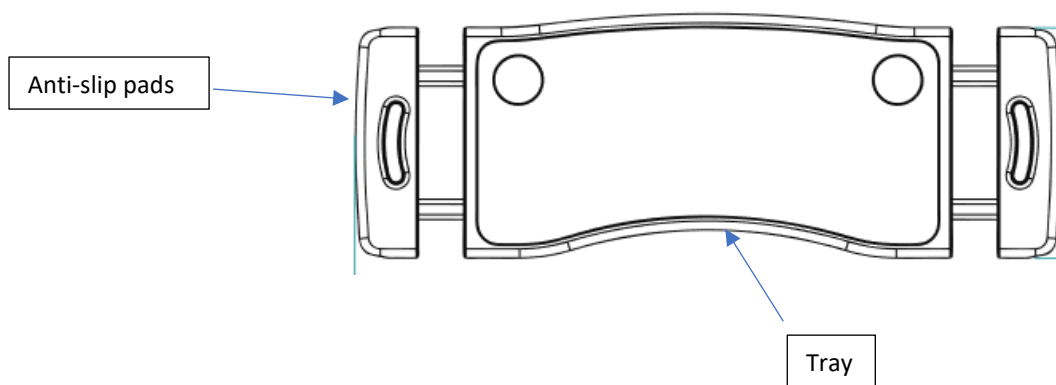
Consult your doctor or health care professional before using a walking aid. He/she will advise and inform you on the most appropriate walking technique for you.

Distributor: this leaflet must be given to the users of this product.

PRODUCT DESTINATION

Bedrail tables are medical devices. This highly practical and ergonomic tray is specially designed to fit over bed rails. Its 2 sides rest on the rails and are fitted with non-slip glides. Ergonomic tray. Equipped with 2 hooks on the underside for storage against the rail.

PRODUCT DESCRIPTION

SAFETY PRECAUTIONS

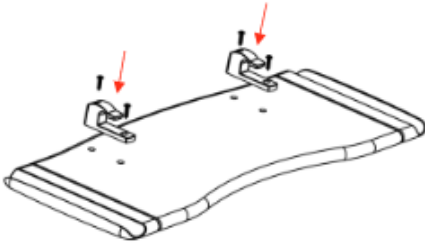
Failure to observe the precautions for use may result in the risk of serious injury.

- Use the product only as described in this manual.
- Do not exceed the maximum operating weight of 10kg.
- Store the product indoors, away from mould and dust.
- Before use, check that all parts are present and in good condition. In the event of damage, please contact your dealer.
- Follow the assembly instructions carefully and make sure that all components are correctly secured before use.
- If any parts are damaged, broken or worn, contact your dealer.

MOUNTING AND USE

- **Installation:** Insert the screws through the holes of the hooks and the board. Then tighten the screws. (Already installed in the factory)

- 1 Extend the rack by pulling out the wings. Place the board on the siderails. (Caution: the grooves on both sides of the board should be placed on the siderails.)

- Safe load capacity: 15kg

- 2 Please hang the table at the head/foot board after use.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the product regularly with lukewarm water, a soft cloth and a neutral detergent.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Dry with a soft, clean, dry cloth.
- Make sure there is no accumulation of dirt on the product.
- Periodically check all screws for wear and strength. Replace all broken, damaged or worn parts immediately.
- Regularly check the operation of the anti-slip pads
- In the event of a malfunction, do not repair the table yourself. Consult an authorised dealer.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Respect the conditions for storing and stowing your product.

- In a dry and temperate place
- Protect your product by packaging from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place too heavy objects on the product components, do not jam anything...).

TECHNICAL SPECIFICATIONS

Overall dimensions	Length 88 to 104 cm (top 70.5 cm) x Width 33.5 cm x Thickness 2.8 cm
Weight	2,3 kg
Maximum weight supported	10 kg

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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