playcare

Care One S

Instrucciones / Instruccions / Instructions  
Argibideak / Instruccions / Instruccions / Instruções  
Instructions / Istruzioni / Anleitungen

Componentes / Components / Components / Osagaiak  
Componentes / Components / Componentes /  
Composants / Componenti / Komponenten

## ES

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Keep your purchase receipt to exercise your rights as a consumer. The company reserves the right to make changes to the product from what is described in this instruction manual.

Malfunctions caused by natural disasters, maintenance by a non-original factory, or self-modifications are not covered by the warranty. This product is intended as a walking aid and is not suitable for individuals who cannot walk independently.

For people with vision problems, dyslexia, or cognitive disabilities, share the instructions with family members.

Before use, ensure that all functions work properly. Stop using it if the walker is damaged or malfunctions, and contact the authorized dealer.

This walker is intended for indoor use and for users with a maximum weight of 100 kg. The maximum load in the basket is 5 kg. Be cautious and use the brake system when walking on a ramp. Do not store in direct sunlight or in hot/humid areas to avoid malfunction or deformation. Do not use on ramps or uneven surfaces in rainy weather or slippery ground. Improper use may cause dangerous situations: do not use the product to carry a person.

Please refer to the Warranty Terms and Conditions at [www.playcare.es](http://www.playcare.es)

**Washing instructions:**

Read the care label carefully.

Do not use solvents or bleach that could damage the materials.

Metal and plastic parts can be hand-washed with warm water and mild soap. Do not use harsh substances.

Dry the cart frame thoroughly after contact with water (or rain) to prevent moisture.

Do not modify or disassemble the frame to avoid damage or malfunction.

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Imported by:  PLAY S.A.
NINGBO NEWKIND HOME CARE CO., LTD.
PLAY S.A.