
carlett.

ENJOY THE ECOMOTION

Lett WALK

Componentes / Components / Components /
Composants / Komponenten / Componenti

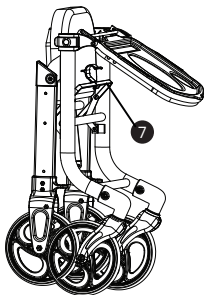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Remember to keep the receipt, because it is your only proof of guarantee if the product is defective.

Any defect arising from misuse of the product by the user, modification, dismantle the product, and the possible normal deterioration of certain components (wheels and textile components) due to wear and tear of the product are excluded from the guarantee.

This walking trolley is deemed to meet the Essential requirements and relevant provisions of EC Directive: Medical Devices Directive (UE) 2017/745. The company cannot be held responsible for any other use made of the product.

Do not move the trolley when a person is sitting on the seat. IT IS NOT a wheelchair.

The maximum weight that the bag holds is 10 kg. Do not overload the bag even when the maximum weight has not been reached.

Take special care when using the product on a ramp or a street with a slope, especially if the trolley is loaded.

Remember to use the handbrake to control the trolley. Take special care when using the trolley on wet or slippery ground. Check that the safety or parking brake is on before using it as a seat.

This product is not a toy. Do not allow children to handle and/or climb on it.

Do not use the trolley to transport pets.

Do not use the trolley to transport babies.

Do not keep the trolley in places that are damp, very hot or exposed to direct sunlight, as this may shorten the life of the product.

This product is used as a walking aid, not suitable for people who cannot walk independently.

Make sure that the handlebar is correctly locked after adjusting the height before using it.

Make sure the basket is empty before folding the trolley and be aware of all the moving parts when folding the trolley to avoid hurting any part of your body.

Be aware that the trolley does not have any lock to fold or unfold the chassis.

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