### IMPORTANT

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### IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using an assistive device. He or she will advise and inform you about the most appropriate use in your case.

Distributor: this leaflet must be given to the users of this product.

### PRODUCT DESTINATION

The Soft shower lift chair gently lifts the user from a seated position. The jack compensates for some of the force required to get up. The seat is height-adjustable. Wide armrests allow you to lean forward to sit down and stand up. The armrests are also padded for a non-slip grip. Fitted with wide, non-slip feet.

### PRODUCT DESCRIPTION

### SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the weight limit of 136kg.
- Read these instructions carefully.
- Check that all components are undamaged and securely fastened before use.
- Suitable for indoor use only.
- Clothes or body parts can be pinched by sitting down, standing up, inserting the toilet bucket and by raising and locking the armrests.
- Prevent unauthorised use, for example by children.
- If any parts are damaged, broken, worn or missing, contact your dealer.
- Check that the feet are fully extended before each use and that all 4 feet touch the ground at the same time, without any play.

- Before each use, check that the system is properly locked (the spring-loaded click must lock). If any parts are damaged, broken, worn or missing, contact the dealer.

## ASSEMBLY AND USE

The products are delivered half-assembled and can be fitted without tools.

### Step 1

- Attach the backrest tube to the frame by pressing the push buttons and sliding the backrest tubes into the tubes under the seat.

### Step 2

- Attach the feet to the frame by pushing the push-buttons into the tubes on either side of the feet.
- Make sure that the push-buttons engage correctly in the holes and that they are stable and set in the same way.

### Step 3

- You can sit down on the seat and start showering.

*Note: the armrests and seat can be folded up.*

## CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Wipe the product dry after each use.
  - Periodically clean the product with a damp cloth and a mild detergent or disinfectant. For example, a mild chloride-based disinfectant can be used.
  - If the locking mechanism does not seem positive or if there is excessive movement in the appliance, remove it from service and contact your dealer.
  - Check the product for cracks and damage, particularly to the backrest and seat.
  - Do not use in environments above 60°C (saunas, for example).
  - If there is a malfunction, do not repair the product yourself. Consult an authorized dealer.
- Checklist :
- To keep the product in good condition, check before each use, and check regularly as follows:

Check the article	Anytime	Weekly
Check the operation of the height adjustment studs	X	
Check the condition of the rubber feet, backrest and seat		X
Check the operation and condition of the armrests	X	
Check the operation of the lifting aid	X	
Check that the supports are fully attached to the edges of the bathtub	X	

## CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

## RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

## STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your product.

- In a dry and temperate place
- Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

## TECHNICAL SPECIFICATIONS

Overall dimensions	W 75/79 cm x D 58/61.5 cm x H. 85/95.5 cm
Seat dimensions	Width 47.5 cm x Depth 42 cm x Height 45/55 cm
Structure	Corrosion-resistant aluminium
Weight	Aluminium 6.5 kg
Maximum authorised weight	136 kg (300 lbs.)

## PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

## WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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