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en This mask is designed, developed and manufactured by **Thuasne, a French group specialising in medical devices** that is ISO 9001/13485-certified.

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en This mask is neither a medical device within the meaning of Regulation (EU) 2017/745 (surgical masks), nor an item of personal protective equipment within the meaning of Regulation (EU) 2016/425 (such as an FFP2-type filtering mask).
- Ergonomic shape
- Polyamide and cotton fibre for optimal continuous wear for 4 hours

Name

AFNOR SPEC S76-001 barrier mask
This mask is for non-medical use (NMU) only and is aimed at personnel working in jobs or tasks involving regular contact with the public.
It protects against the emission or dispersal of droplets of saliva or upper airway secretions that could contain transmissible infectious agents.

Precaution

To protect your health and that of others, it is very important to comply with the product information leaflet.
If you are ill, this mask is not suitable; consult your doctor for advice.
This mask is not intended for healthcare professional.

Washing instructions

Your Thuasne barrier mask is reusable and must be washed with care:
- Wash the mask at 60°C on a complete wash cycle lasting at least 30 minutes.
- Wash the mask every day, after each use.
- Tumble-drying and ironing recommended.
- Store at room temperature, in original packaging.

This mask can be washed 100 times*.
A washing table can be downloaded at:

<https://fr.thuasne.com/fr/masque>

Masks must be completely dry before use.
Once the maximum number of washes has been reached, the mask must be discarded:
- using the specific procedure for the disposal of infectious clinical waste,
- in a carefully sealed plastic bag if disposal via the infectious clinical waste procedure is not possible.

Composition: Polyester, cotton, elastane, contains latex
Size: adult.

Contraindications
- Known allergy to any of the components.
- Do not place the product in direct contact with broken skin.
- Severe respiratory insufficiency.

Type of product :
"Category 1 mask for non-medical use only in accordance with the information note of 29 March 2020".
Material for which the performance has been measured by the French Defence Department's chemical, biological, radiological and nuclear risk control unit laboratories, 5 rue Lavoisier, 91710 Vert-le-Petit (IFTH tests ref. 20-01731) and which tolerates 100 washes
- filtration efficacy for 3 µm particles emitted > 90%**
- air permeability for a pressure drop of 100 Pa > 96 L.m⁻².s⁻¹**

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(en) This barrier mask is not a replacement for application of protective measures (regular hand-washing, social distancing, reduction of contact with other people). It adds a physical barrier enabling the user to protect his/her environment from the projection of particles. It can also provide the user with limited protection against infectious agents.

It is essential to put your mask on properly to ensure good protection.

Wash your hands with soap and water or alcohol-based sanitizer before putting on a mask.

Make sure the mask is clean and dry before you put it on.

- Unfold the mask ❶, with the blue edge upwards. The OUTSIDE is identified by the Thuasne logo.
- Place the elastic straps over the head. First the bottom strap, then the top strap. Apply the mask so that it covers the nose, mouth and chin. Adjust it carefully to fit the face as closely as possible ❷ ❸.
- Adjust the nose piece at the top of the mask ❹.
- Make sure the mask is properly in position by checking the seal. To do this, cover the mask with a plastic film; when you breathe in, the mask should flatten against your face ❺. Check that there is no air blown into the eyes when you breathe out hard.
- For your comfort, we have designed the straps to be modular. If you prefer to wear the mask with the straps behind your ears, cut the straps at their centre, tie the straps at the same side together and place around the ears. ❻
- Once the mask is in place, breathe normally.

Always check that the mask is properly adjusted and that it covers your mouth and nose. It is recommended that this mask be worn over bare skin; beards may reduce the filtration efficacy to outside the target range.

The mask can be worn for 4 hours continuously between each wash. When wearing a mask, avoid touching it and do not move it.

Wash your hands with soap and water or alcohol-based sanitizer each time you touch a used mask. If you need to eat or drink, change your mask.

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