# **SILLÓN DE MASAJE ELEVADOR ECO-8198UP**

**ESPAÑOL  
FRANÇAIS  
ENGLISH  
DEUTSCH  
NEDERLANDS  
PORTUGUES  
ITALIANO**

# SILLÓN DE MASAJE ELEVADOR ECO-8198UP

## ESPAÑOL

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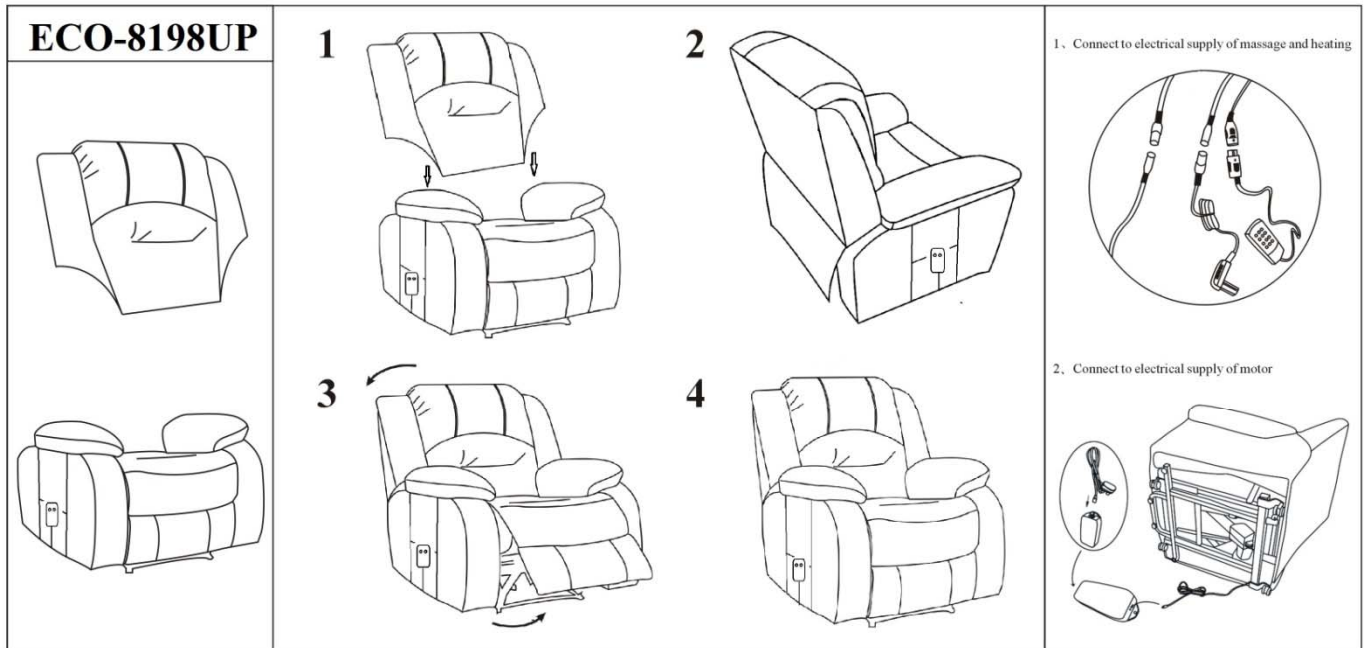
ATTENTION: MAX PERSON WEIGHT USAGE IS 125KG  
RECOMMENDEDE DOMESTIC USE

## WARNING

1. The fittings for our raclner chairs should be tightened every three months.
2. the chair should not be assembled if any fittings are missing from pack.
3. Do not sit on arms or stool.
4. Do not use power tools as they can damage threads

5. To clean chair wipe down with damp cloth. use water only
6. Do not put any part of chair in direct sunlight
7. Do not places near heat
8. To mave chair, lift, do not drag as this will damage chair

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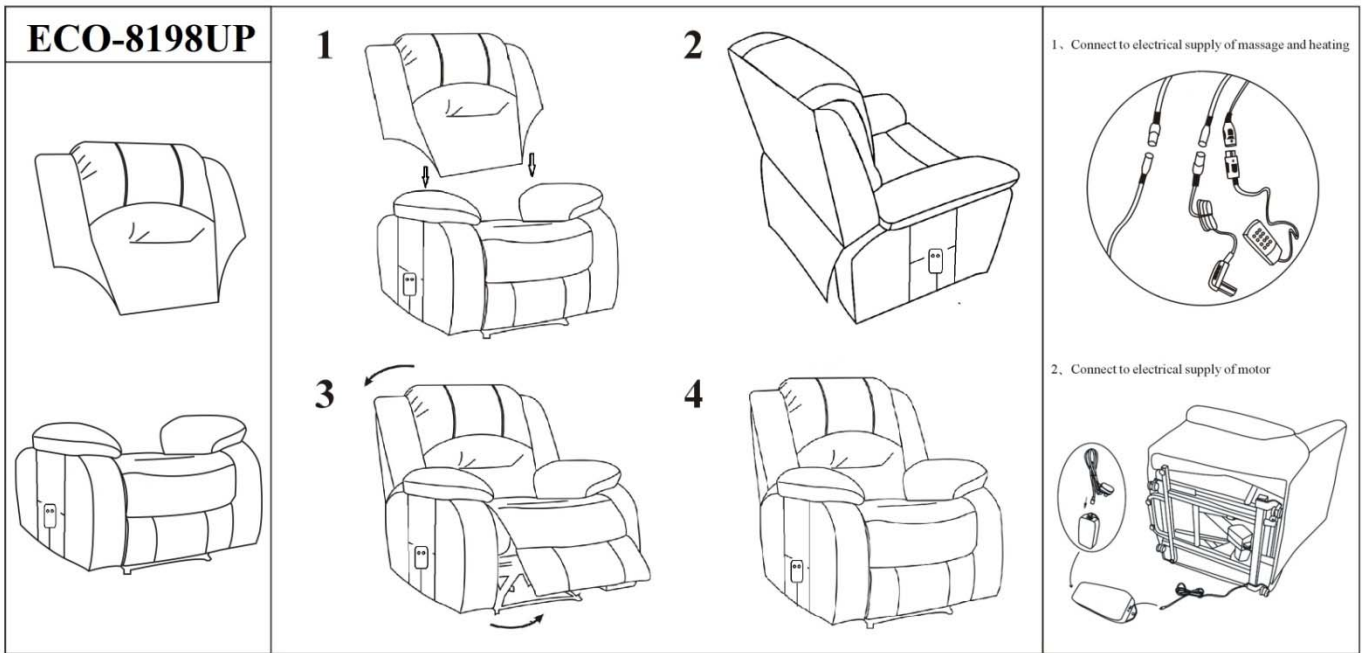
- Do not use the machine in damp and dirty places to avoid electric shock or product damage.
- Do not use it in places where the temperature is over 40°.
- Do not store the machine outdoors.
- Always make sure make sure to use on leveled surfaces.

#### CAUTION!!!

- To remove from lower temperature condition to warm circumstance, please wait for one hour before using it, to avoid any freezing onto the machine. Otherwise the machine could get damaged.
- Check the rated working voltage.
- Do not attach or detach plug with wet hands to avoid electricity leaking or product damage.
- When detaching the plug, please hold the insulated plug instead of the power cord.
- The setted time functions can make the machine stop working automatically. Make sure to shut the power cor off after using the device.
- Do not place anything heavy onto the power cord.
- If any damage occurs on the power cord, do not use the product any longer, please turn off the power and detach the power plug and contact the service center for maintenance or replacement.
- Do not discover the leather cover to touch any interior components, to avois electrical shock.
- Should any problem happen, contact the nominated service center for maintenance.
- Never dismantle the machine yourself.

#### **¡WARNING!**

**This device produces heat, when in use. Be careful.**

**WARNING**

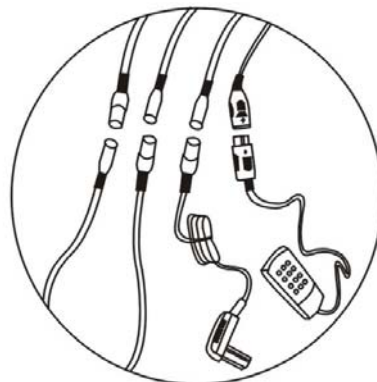
- 1. The fittings for our raclner chairs should be tightened every three months.
- 2. the chair should not be assembled if any fittings are missing from pack.
- 3. Do not sit on arms or stool.
- 4. Do not use power tools as they can damage threads

- 5. To clean chair wipe down with damp cloth. use water only
- 6. Do not put any part of chair in direct sunlight
- 7. Do not places near heat
- 8. To mave chair, lift, do not drag as this will damage chair

Any warranties offered with this chair will be deemed null and void should you not follow our recomendations

**Step 1:** let the backrest slip until it interlocks on the base.

**Step 2:** connect all the plugs as in the picture. Do not force this process, once each plug is only connectable to its correspondant.

Connecting massage back and seat..

Connector foot massage.

Transformer connector

Wireless control button

## **ARMCHAIR MASSAGER SPECIFICATIONS**

(Adapter Power input) Voltage : 230V-50HZ

Output Power: DC12V

## **REMOTE CONTROL**

**MODE:** mode selection

**ON/OFF:**

**1-9:** indicator of 9 massage modes

**INTENSITY:** / intensity selection from 1(-) to 5 (+)

**ESPALDA-LUMBARES-ASIENTO -PIERNAS:** massage area selection.

**HEAT/CALOR:** select the heat on the back

## START

- **START, ON/OFF:** plug the cable in and press the side button

## MASSAGE FUNCTIONS

- button: **MODE**

Pressing this button you may choose between 9 massage modes. Pressing it repeatedly you can change from mode 1 to mode 2 and so on..

- button: **INTENSITY**

intensity selection from 1(-) to 5 (+).

The automatic system is the one that is the most recommended by our engineers and physiotherapists. Close to a professional massage, the seat offers automatically a massage going from the gluteus muscles to the shoulders.

- buttons: **ESPALDA-LUMBARES-ASIENTO -PIERNAS:**

To select the massage area

## PROGRAMM 1: RELAX TOTAL

For a muscular relaxation of the whole body. Specially recommended for people suffering from insomnia. Use it before going to sleep.

## PROGRAMM 2: ANTI ESTRÉS

Ideal for after a large working day, to eliminate all the accumulated stress.

**PROGRAMM 3: RELAX PARCIAL**

This program relaxes certain areas, giving a massage only in the chosen areas. Ideal to eliminate muscular stress and articular pain

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Through a monitored massage system, it activates the whole circulatory system.

**PROGRAMM 5: CONFORT PLUS**

This program alternately relaxes different massage areas to cause greater relaxation

**PROGRAMM 6: EXTRA RELAX ACTIVATION**

Program activation of blood supply combined with active muscle relaxation

**PROGRAMM 7: TOTAL SHIATSU**

Active points covered by Shiatsu massage

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Through a monitored massage system, it activates the whole circulatory system.

**PROGRAMM 9: SLEEP POWER**

Distributes pressure points for a more relaxing massage allowing you to rest better

*USB Function And LED Light*

*Reclining*

*Lift Function*

**USB PORT to recharge electrical devices; mobile phones or tablets**

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## **GUARANTEE**

### **CONDITIONS:**

- A) This product guarantee will be accepted by ECOGEST TRADE S.L., provided that it is correctly completed and the attached card has been stamped and send to ECOGEST TRADE S.L.
- B) The responsibility of ECOGEST TRADE S.L. C/ En Proyecto 14, 46190 Ribarroja de Turia, Valencia (Spain) it is limited to the repair of the articles that has some deficiency of technical character or of manufacture
- C) All repairs which are carried out under the present guarantee should be performed by an establishment which is duly authorized by ECOGEST TRADE S.L.
- D) This guarantee will not be considered valid in those cases which result from the misuse. Incorrect application or manipulation of the product.

Validity of guarantee for 2 years from the delivery date.

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