

# Sliding bath bench Item no. 302065

## User manual



**MOBILEX A/S**  
Grønlandsvej 5  
DK-8660 Skanderborg  
Denmark  
[www.mobilex.dk](http://www.mobilex.dk)

## 1. General information

Dear customer

This user manual contains a description of the medical device and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

### Intended use of the device

Mobilex Bath Bench is medical device intended only to be used by patients that because of invalidity need support during entering or leaving the bathtub and during washing. It makes possible for the patients to pass elevated bathtub's side walls with or without the assistance of a helper (depending on user's condition). Physically handicapped persons can use it as stable support and easy access to a bathtub and all human body parts during washing. The device should be used inside bathrooms only. Bath Bench can be maximally loaded with 130 kg.

Any other use then intended is strictly prohibited

By questions please take contact to your dealer or directly to Mobilex A/S.

### Mobilex A/S

Grønlandsvej 5  
DK-8660 Skanderborg  
Denmark

Phone: +45 87 93 22 20  
Telefax: +45 87 93 17 77  
E-Mail: info@mobilex.dk

## 2. Safety requirements

- Make sure that this user manual is read by all persons using the product. Damages, caused by the fact that the user manual has not been followed, will not be accepted.
- Bath Bench must not be loaded with more than 130 kg
- Use the Bath Bench only for the purpose, that it is intended for.
- Use Bath Bench only in good technical condition.
- Avoid making constructive changes on the Bath Bench, unless you have the producers written accept.
- Before first use adjust the height of Bath Bench's legs to fit it to the height of your bathtub's walls and bottom.
- Adjust the legs, so the seat is in horizontal position
- If you detect any damages, please contact your dealer immediately.
- We recommend to use help of an assistant for getting on and out of the bath bench.
- Use Bath Bench only for seating. Do not stand on the Bath Bench.
- When applying Bath Bench in to a bathtub - watch carefully for all four legs to lie flat, stable and parallel on the top of floor and bathtub's bottom.
- Follow the safety information on product label.
- Always lock seat in position after sliding in or out of the bathtub. Make sure the seat is locked before attempting to sit on it or to get up of it.


## 3. Description

### Product description

Mobilex Bath Bench set consists of base frame made of aluminum, plastic seating pad and backrest. The four foldable legs of base frame are covered with protective rubber feet to be lied down on the floor (2 narrow feet) and bathtub's bottom (2 wide feet). Bath Bench is equipped with safety handle and removable soap holder. For transportation purposes Bath Bench comes seaparted in to few elements which have to be assembled prior first usage:

- 1 main frame with seating pad and foldable legs

- 1 backrest
- 1 red, plastic safety handle
- 2 red plastic wingnuts for handle assembly
- 1 soap holder
- 1 user's manual

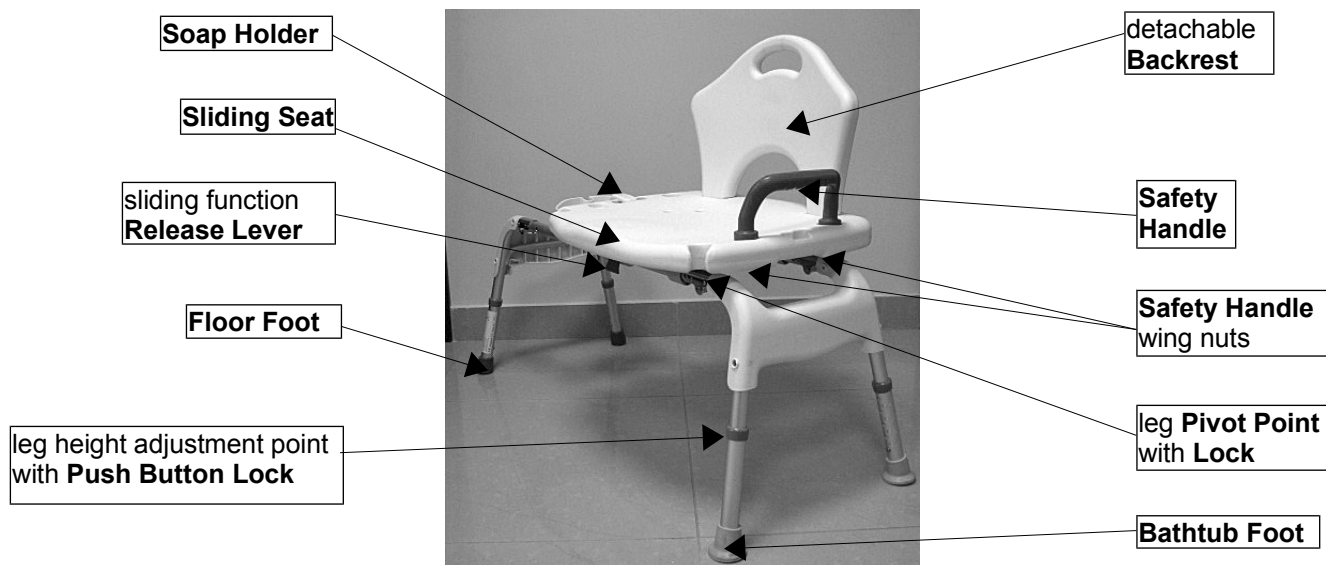

### Assembly & folding description

- Check with the list from above if nothing is missing. If you'd discover at that stage that some part is missing or damaged, do not continue assembly process. Contact your supplier of the product right away.
- Take main frame and open legs by pulling them to the side until you'll hear a "click" sound of legs safety lock buttons. Repeat the same for the other side of the bench.  
**WARNING!** Always before usage ensure that legs are fully opened and locked in that position (as shown on picture above)
- Put the backrest in to it's place by pressing it in to 2 dedicated rectangular holes located at the back of the seat pad. Check if the backrest went all the way and is locked in it's place safely.
- Place safety handle's ends in to 2 holes on the side of the seat pad (as shown on picture above). Use 2 red wing nuts to tie down the handle to the seat by applying them from the bottom.  
**INFORMATION!** You can assemble handle on either right or left side of the seat. Your choice depends on conditions of your bathroom and your personal preferences. For safety reasons it is the best to attach handle on that side of seat which is located at the most outside when bench is sled out of the bathtub.
- Place soap holder at one of the sides of the seat pad by pressing holder's tail in to dedicated hole. You can choose the side of bench where you prefer to have holder attached.
- Adjusting the legs. Legs should be assembled in the way, where 2 wider rubber feet will be located inside of the bathtub and 2 narrower feet will be standing on the bathroom's floor. Legs are preassembled in factory, but if you want to stand your bath bench in opposite direction that it comes from factory preset, you can easily take away the bottom part of legs and switch their sides.
- Legs are equipped with push button locks. Simply press on the button located on each leg, and pull the bottom part of the leg. In this way you can switch the legs' sides or adjust their length.
- Check the difference between floor level and bathtub's bottom level which usually is located higher. Adjust the legs in the way, where bathtub's feet are higher than floor feet. The difference in their length should be the same as the difference between floor and bathtub's bottom level. Then bath bench placed correctly should be leveled. The leg length should also be adjusted so the seat is free to slide and hangs over the bathtub sidewall. The seat's frame should not touch bathtub's top.

- To fold the bath bench press push buttons located in front and in the back of main frame at each leg's pivot point and fold the legs. To limit the bench's size you can also detach handle, soap holder and pull out the backrest if necessary.

#### Usage description

- Place the bath bench so 2 legs (right or left) are placed inside the bathtub and 2 other legs are placed on the floor.
- Sit on the seating pad. Then pull and hold the red lever located under the seat on it's right side. The seat lock will be released.
- By changing body balance and pushing with your feet slide the seating pad with your body towards the bathtub.
- When the seat will stop release the lever. The lock should "catch" and the seat should be secured in position. If the seat isn't locked, make small moves of it untill you'll hear the "click" sound of safety lock.
- To get out of bathtub repeat above steps. The direction of movement will be oposite to previous one.

#### Transport recommendations

If it is necessary, device can be easily folded and transported

#### Accessories

There are no accessories provided for this device.

## 4. Technical data

Mobilex Bath Bench	302065
Overall width	107 - 111 cm
Overall length	49 - 53 cm
Overall Height (adjustable)	89,6 - 99,5 cm
Seat width	50 cm (with handle) / 57 cm (without handle)
Seat depth	39,5 cm
Under seat height (max. height tube edge)	45,5 - 55,5 cm
Weight of the product	6,5 kg
Maximum load	130 kg
Back Height	33 cm

#### Product label

The product label is located on frame in the back of the device




	<b>Sliding Bath Bench</b>
MOBILEX A/S Grønlandsvej 5 DK - 8660 Skanderborg Tel: +45 87 93 22 20 www.mobilex.dk	 ↓ Max. 130 kg
 <b>Produced</b> <input type="text"/>	<b>Serial no.</b> <input type="text"/>

Image of the product label (not original)

## 5. Maintenance & cleaning

Bath Bench is basically maintenance free.

Although it is recommended to check always before use if the anti-slip rubber feet aren't loose or torn and if they lie down on the bathtub's bottom and floor in stable position. Check also if the legs are locked in opened position and if the backrest is attached to seat correctly. In case of any doubts concerning technical condition of your Bath Bench stop using it and contact your dealer.

Normal dirt on metal and plastic parts can be removed with standard household cleaning agents. Check the specific product information and follow it during cleaning. Use only commercial cleaning products which are suitable for cleaning and disinfection (no solvents or abrasives).

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to let check the product at regular intervals by a professional

## 6. Most common problems and solutions

<i>Symptoms</i>	<i>possible cause</i>	<i>what to do?</i>
The seat is not in horizontal position	Legs are not adjusted to correct height	Readjust the legs length so the seat is in horizontal position when bench is placed in bathtub.
Back rest is not stable	Backrest is not pressed all the way in to dedicated holes	Press backrest all the way in to it's place.
Safety handle isn't stable	The wing nuts under the handle aren't fixed	Tie the wing nuts.
The frame isn't stable	Legs are not fully opened and locked.	Pull the legs outward until you'll hear the "click" sound of their safety lock.

## 7. Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling centre.

## 8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

## 9. Declaration of Conformity (CE)

The company Mobilex A/S, Grønlandsvej 5, 8660 Skanderborg, Denmark declares that the product

### Sliding bath bench Item no. 302065

is designed in accordance with the technical documentation and made of which conforms to the following basic requirements:

- **EU Council Directive 93/42/EEC for medical equipment, Annex I of the June 14, 1993**

Lars Nygaard  
Managing Director  
Mobilex A/S



Skanderborg, the 01.02.2010