GEEMARC

CLEAR SOUND™

HEAR THE DIFFERENCE

# PhotoPHONE

English

Français

Deutsch

!!!User manual to read carefully before start to use!!!

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**CAUTION:** this telephone is specifically designed for the people with hearing difficulties and is provided with a handset earpiece volume control for individual requirements. Due care must be taken by all users that the handset earpiece volume control is set to the lowest level acceptable by each respective user. Care should therefore be exercised to ensure that any naive users are familiar with this requirement.

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## INDEX

<b>Description</b>	<b>7-8</b>
<b>Setting up</b>	
- Connecting the telephone	9
- Ringer settings	9
- T/P switch	11
- Time break recall switch	11
- Wall mounting	11
- Desk support	13
<b>Volume and tone adjustment</b>	
- Receiving volume and tone	14
- Sending volume level adjustment	15
<b>Using the phone</b>	
- Answering incoming call	16
- Making a call	16
- Last number redial	16
- Recall, * & # buttons	17
- Mute button	17
- Storing telephone numbers	18
- Dialling stored number	19
<b>General information</b>	<b>20</b>

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
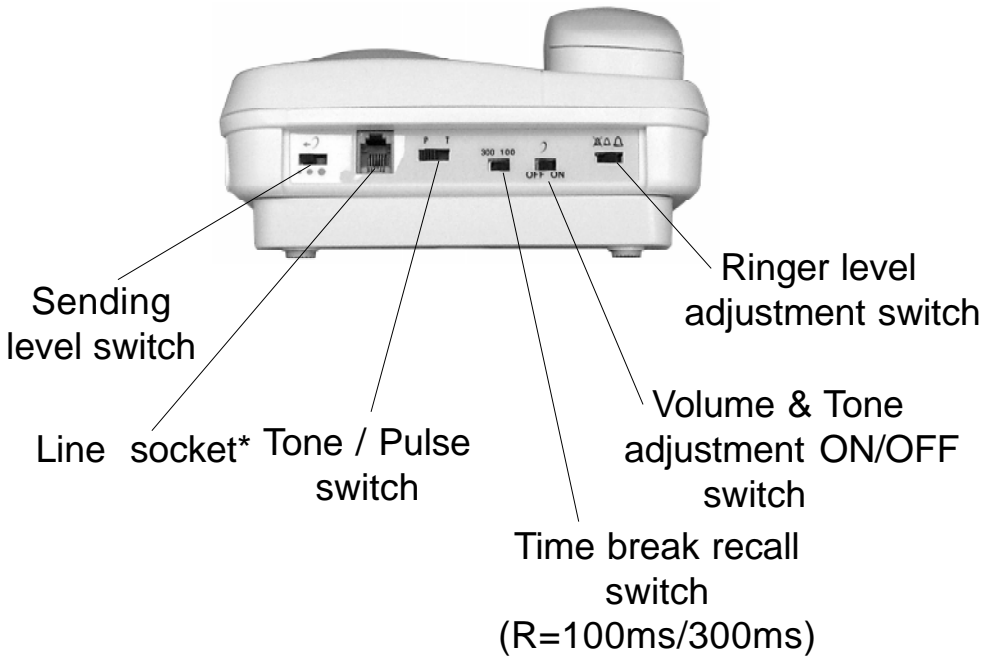
# Description

# Description

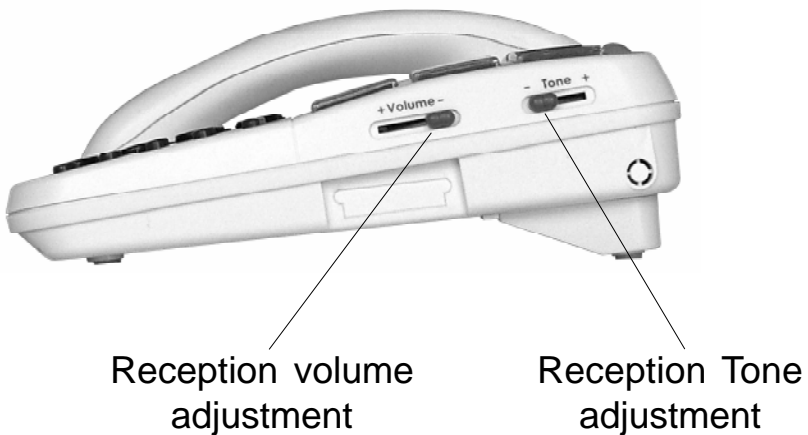(\*) Classified TNV-3 according to EN60950 standard.

## 1 - CONNECTING THE TELEPHONE

- Connect the handset cord.
- Open the battery compartment.

Insert 4 alkaline batteries without mercury (Type AAA 1.5V, not supplied) in the compartment (\*). Batteries are required if you wish to increase the ringer level up to 10 dB.

Snap the battery door back in place.

- Connect the line cord plug to the line socket on the base (\*).

Connect the other end of the line cord to the PTT socket. (\*)

## 2 - RINGER SETTINGS

A switch located on the upper side allows the ringer to be switched HI,LOW,OFF:

 : Off,

 : Low,

 : Hi

# Setting up

Your phone comes with the ringer switched to «High» and the melody set to «6»:

	Low	Mid	High
Melody 1	1	2	3
Melody 2	4	5	6 (by default)
Melody 3	7	8	9

If you don't like the ringer melody, you can choose a new one by changing the settings as per instructions below:

- Lift the handset.
- Press  button.
- Press # button.
- Press any of the 1 - 9 dial keys.

The new ringer settings will be activated with the next incoming call.

**Note:** The ringer level will be increased up to 10 dB if you insert batteries.

(\*) Classified TNV-3 according to EN60950 standard.

### 3 - TONE/PULSE SWITCH (T/P)

The Tone/Pulse switch is factory pre-set to Tone position (T). In UK, all telephone exchanges now use Tone dialling. If your phone does not dial out, it is probably being used from an older private switch-board (PBX). In this case, the Tone/Pulse switch must be moved to Pulse.

### 4 - TIME BREAK RECALL (TBR) SWITCH

For UK, the TBR switch should be set to 100 mS. It can be switched to 300 mS for other locations or PBX's.

**Note: If you are using your CL1100 with a hearing aid, set your hearing aid to “T” position.**

### 5 - WALL MOUNTING

To wall-mount your telephone, slide out the clip, turn and slide back into the wall mounting position

# Setting up

(located in the base unit, just below the ear piece). This will ensure that the handset stays firmly in position.

Step 1

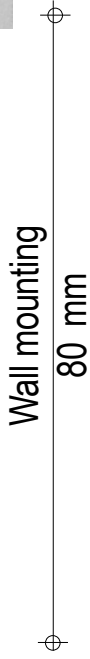
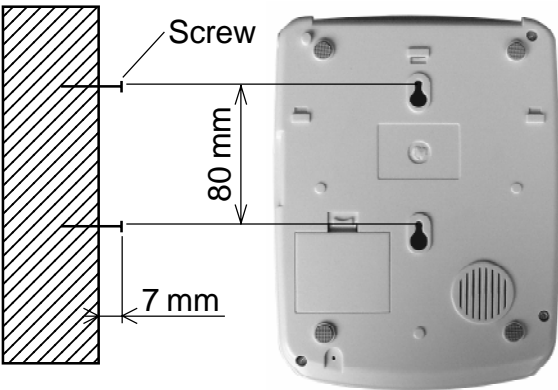
Step 2

Step 3


Make 2 vertical holes in the wall 80mm apart, insert wall plugs and wall mounting screws. The screws should protrude from the wall by 6-7 mm. Place the phone onto the screw-heads and slide down to secure.

## 6 - DESK SUPPORT

The phone can be used on the flat position or raised with the desk support. The desk support will give you the best viewing angle.

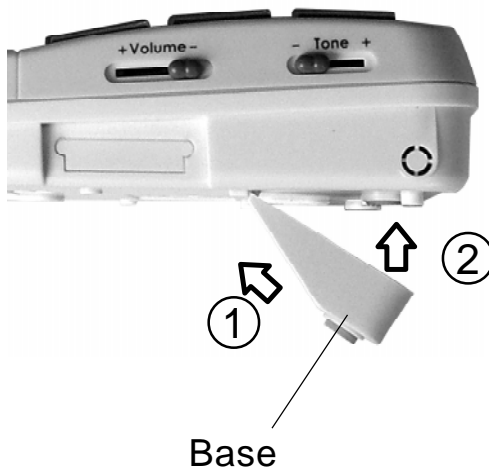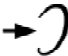
To fit the desk support, insert the two upper hooks into the openings provided and push the support firmly home.

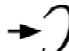
# Tone and volume adjustment

## 1-RECEIVING VOLUME AND TONE ADJUSTMENT

You can adjust the receiving volume and tone according to your own hearing requirements.

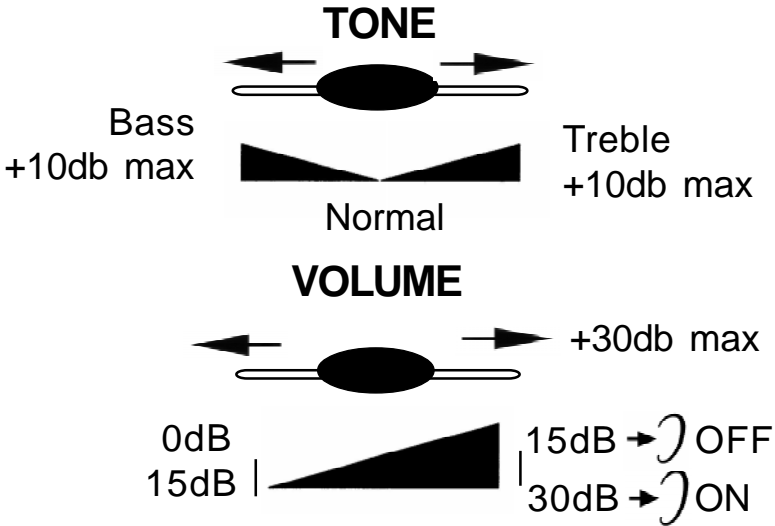
The button  allows extra volume and tone functions to be activated or deactivated during the conversation.

Tone and volume adjustment (15dB to 30dB) is only activated when the switch  located at the upper side of your phone is set to **ON**. When this button is set to **OFF** only the volume can be adjusted from 0dB to 15dB.

When the function is activated, the red light of extra receiving and tone function button  is lit.


**Note:** When the tone (treble), earpiece volume and sending levels are set to the maximum, whistling may occur. If whistling occurs the sending level settings should be decreased with the switch 

Adjust the **TONE** and **VOLUME** buttons to fit the levels to your own hearing requirements.

## 2 - SENDING VOLUME LEVEL ADJUSTMENT

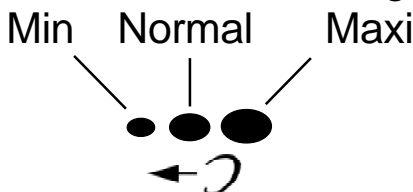If your voice is quiet or called parties have difficulty hearing you, you can increase the sending level volume with the switch located on the upper side. You can also decrease the sending volume if required.

# Using the phone

## 1 - ANSWERING INCOMING CALLS

- When an incoming call is received the phone will ring and the incoming call indicator will flash. To answer the call lift the handset and speak.
- On completion of the call, carefully replace the handset in the cradle to release the line.

## 2 - MAKING A CALL

- Lift the handset.
- Await the dial tone and dial required number.
- On completion of the call, carefully replace the handset in the cradle.

## 3 - LAST NUMBER REDIAL

- Lift the handset.
- Await the dial tone and press  button.
- A previously dialled number will be automatically redialled (not in use for memory numbers).

## 4 - RECALL, \* & # BUTTONS

These are used with the new services provided on digital exchanges. For details please contact your network operator.

## 5 - MUTE BUTTON

If you wish to speak privately to someone else in the room without your caller overhearing, press and hold the  button. You will still be able to hear the callers but they cannot hear you during the mute operation. To resume normal conversation, release the  button.

## 6 - STORING TELEPHONE NUMBERS

Your telephone can store 9 direct memories. To store a number please follow instructions below:

- Lift the handset.- Press → button.
- Press **M1**, **M2**, ... **M9** to locate the number.
- Dial the number.
- Press again → button.

Numbers will be lost after few hours if you disconnect your phone. Entering a new number in the same location will automatically erase the previous number.

You can insert your contact's photo here.

## 7- DIALING STORED NUMBERS

- Lift the handset and await the dial tone.
- Press the appropriate location button M1,M2 ...M9 and the number stored in this location will be dialed automatically.

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# General informations

## RECYCLING DIRECTIVES

The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products in end of life are recycled in the best way.

When this product is out of order, please do not put in you domestic waste bin. Please use one of the following disposal options :

- Remove the batteries and deposit them in an appropriate WEEE skip.
- Or hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.

## GUARANTEE

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of one year. During this time, all repairs or replacements (at our option) are free of charge. Should you experience a problem then contact our helpline or visit our website at [www.geemarc.com](http://www.geemarc.com). The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative. The Geemarc guarantee in no way limits your legal rights.

**IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM**

**Please note:** The guarantee applies to the United Kingdom only.

**Declaration :** Hereby Geemarc Telecom SA declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 1999/5/EEC and in particular article 3 section 1a, 1b and section 3.

**Telephone connection :** Voltages present on the telecommunication network are classified TNV-3 (Telecommunication Network Voltage) according to the EN60950 standard.

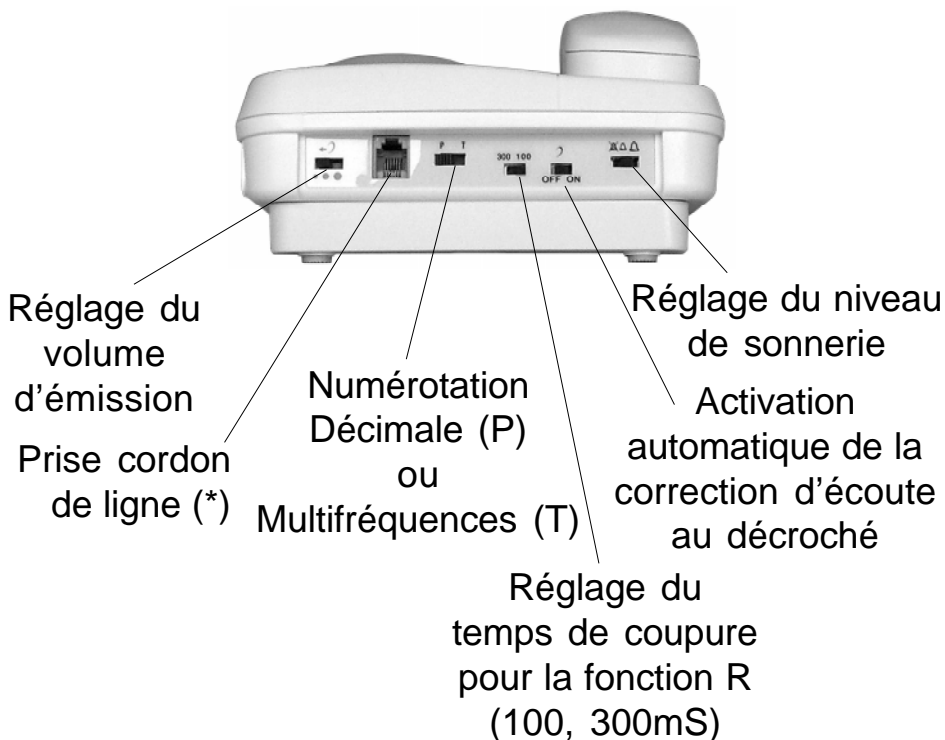
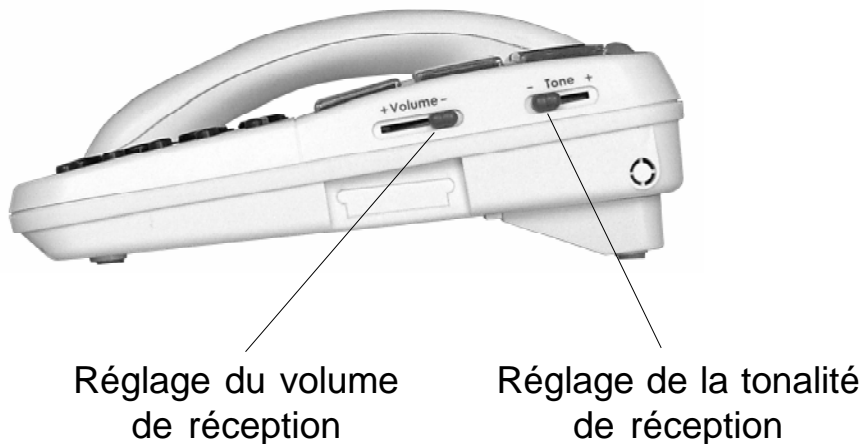
The telephone does not operate if the line current is lower than 18 mA.

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