



Step Stool with Handrail



VM862

Fixing and Maintenance Instructions

INTRODUCTION

Thank you for purchasing the Step Stool with Handrail from Aidapt.

Please examine your product for any visible damage before using/installing. If you see any damage or suspect a fault, please do not use, but contact your supplier.

INTENDED USE

The Step Stool with Handrail is ideal for those hard to reach places. It is a versatile step that can be used in the bathroom, kitchen or other areas around the home. It consists of an anti-slip rubber surface, rubber ferrules and a rubber handrail for additional grip.

ASSEMBLY INSTRUCTIONS


1. Attach metal poles to step base using the bolts and locking nuts provided. Only tighten half way (*fig. 1*)
2. Attach handle to assembled section using the fastening buttons (*fig. 2*)
3. Ensure the buttons fully pop out of each hole (*fig. 3*)
4. Fully tighten the bolts and locking nuts (*fig. 4*)



(fig. 1)



(fig. 2)

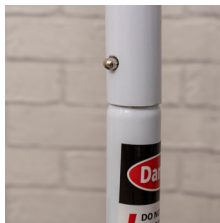

(fig. 3)



(fig. 4)

HOW TO USE

When using the step, please ensure you only step on the centre.

WARNING

- Maximum weight load limit: 150kg (23st 8lbs)
- **DO NOT** exceed maximum weight load
- **DO NOT** use near hot surfaces, cookers or hot water in case of fall
- **DO NOT** stand on crossbars at any time. Risk of injury
- **DO NOT** jump on step
- **DO NOT** lean from step when in use
- This is **NOT** a toy; children should be supervised around this item
- Not recommended for use by anyone who needs assistance walking
- **DO NOT** use on uneven or unstable surfaces
- Periodically check to ensure all fixings are secure, and tighten if necessary

CLEANING

The Step Stool with Handrail should be cleaned using a non-abrasive cleaner or mild detergent with a soft cloth only. Abrasive cleaners or abrasive cleaning pads may seriously damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment.

If to be disinfected by heat, one of the following three temperatures and duration of exposition can be used:

- a) a temperature of 90°C for 1 minute
- b) a temperature of 85°C for 3 minutes
- c) a temperature of 80°C for 10 minutes

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

TECHNICAL INFORMATION

Width 460mm
Depth 265mm
Height 860mm
Net Weight 2.9kg

MAINTENANCE

Please check the Step Stool periodically for any signs of damage and if in doubt contact your supplier.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

A copy of the instructions leaflet can be downloaded from our website.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX

Telephone: +44 (0) 1744 745 020 Fax: +44 (0) 1744 745 001

Web: www.aidapt.com

Email: sales@aidapt.co.uk

