

## POTTY CHILDREN'S TOILETSEAT

Articlenumber: 144971, 144972

**Intended use:** The potty toilet seat is mounted on an adjustable rack, which gives a lower seatheight and therefore a more secure feeling than sitting on a conventional toilet seat. The child can sit upright with easily adjusted reins.

**Delivery Control:** Check that the product is intact and that the rubbertip is mounted on all legs. Do not use product if damaged.



### Safety precautions:

- The product should not be used otherwise than as intended.
- The toiletseat shall be placed on a flat surface where all four legs are firmly placed on the floor.
- Any sign of wear in the heightadjustment or in any other way, the product should be taken out of use and sent for service and maintenance.
- The rubbertips should be controlled in a regular basis. Damaged or worn rubbertips could cause damage to the floor and/or cause deteriorated grip.
- Damaged or worn rubbertips shall be exchanged. New can be ordered from Swereco.
- Use a harness/belt to help the child sit in an upright position.
- Never leave your child unattended on the toilet seat!
- Maximum user weight is 50 kg.

**Symbols:** Explanation of the symbols on the product and directions of use.



Max 50 kg

Maximum userweight 50 kg.



Can be washed with soap alkaline, pH 5-9.



Autoclave max 85°C for max 3 minutes.



Visual control



Product complies with EU directive 93/42/EEG och 2007/47/EG for medical devices.



144971



144972

### Technical specification

Art. Nr: **144971**  
**Toiletseat 1**

#### Size seat:

Depth: 22 cm  
Width: 20 cm  
Height\*: 23 cm

#### Size hole:

Depth: 16 cm  
Width: 12 cm

Adjustable height:  
19,5-28 cm

Max width chair:  
38 cm

Max width frame:  
41 cm

Weight: 3,0 kg

Art. Nr: **144972**  
**Toiletseat 2**

#### Size seat:

Depth: 29 cm  
Width: 23 cm  
Height\*: 35/25 cm

#### Size hole:

Depth: 18 cm  
Width: 14 cm

Adjustable height:  
22-30,5 cm

Max width chair:  
37 cm

Max width frame:  
42 cm

Weight: 3,8 kg

Material: ABS

\*Height from seat up to position of the seatbelt.

Manufactured by:

**Swereco AB**

Vagnvägen 4

SE-364 42 Lenhovda

Tel: +46 (0)10 330 23 10


Fax: +46 (0)8 754 30 80

www.swereco.com

info@swereco.com

### Directions of use:

1. Adjust the toilet seat to proper seat height by pushing the button.
2. Check the seat height adjustment button on a regular basis. If the button is loose, the use of the toiletseat must be discontinued immediately.
3. Check the rubber tips before use. Damaged or worn tips shall be replaced.
4. Put the removable potty in its place.

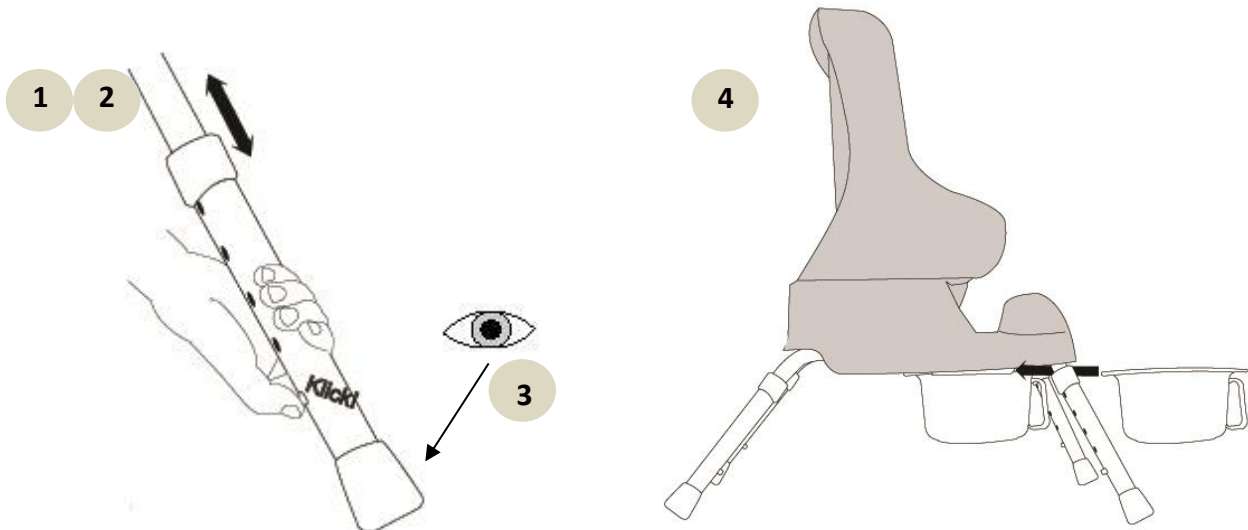

**Care instructions:** Clean regularly if necessary (min 1 time/month) with soap alkaline, ph 5-9. If the toiletseat is given to another user the product shall be disinfected, autoclave max 85° for max 3 minutes, or wipe with 70% isopropanol.

### Spare parts:

Art. nr: 120700, Rubber Tip Hygiene, 21-22 mm  
Art. nr: 14497115, Potty  
Art. nr: 14491040, Harness  
Art. nr: 14497110-S, Leg incl. Rubber Tip for 144971  
Art. nr: 14497210-S, Leg incl. Rubber Tip for 144972

314971