drive
MEDICAL DESIGN & MANUFACTURING

www.drivemedical.com

diamond rollator
andador diamond
déambulateur





EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

assembly instructions

1. First remove all contents from carton: backrest, handle with tube, rollator frame and basket.
2. Open frame by pushing down on sling seat on both sides until the rollator is open all the way.
3. Turn the rollator upside down so both hand grips are on the floor and the front and rear caster receptacles are facing upwards.
4. Insert the back caster by inserting “A” into “B” with the casters turned upwards (figure 1). The back caster will be positioned properly when the brass release buttons “click” through the frame.
5. Insert lock release knob (figure 2) to lock caster assembly into place.
6. Repeat steps 4 and 5 to install other back caster.
7. To install front casters, insert caster leg into front caster journal until a “snap” is heard (figure 3).
8. To adjust handle height, depress brass lock pin on frame (figure 4) and either pull or push handle to desired height. Be sure to repeat process for other handle and ensure both lock pins are in the same position.

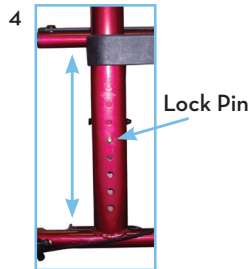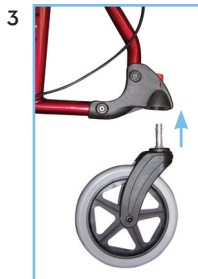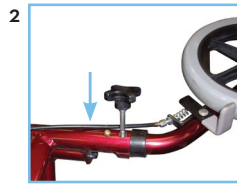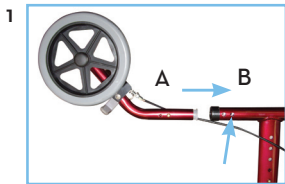safety precautions

1. Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
2. Be sure that both knobs are fully tightened and are at equal heights. Test Rollator for stability before applying full weight to hand grips.
3. Test the loop locks before operating the Rollator. When wheels are locked (see illustrations for locking and releasing locks below), lock bar on rear wheels should prevent wheels from turning. If the wheels turn while brakes are locked, please take unit back to dealer for adjustment. When locks are released, the wheels should turn freely.
4. Always lock loop locks before sitting and do not use seat when unit is on an incline or uneven ground.
5. Replace tires when significantly worn or damaged.
6. Do not have anyone push you while you are sitting back wards on the rollator. You can safely be pushed only when you are sitting forward and you feet are correctly positioned in the footrests.

loop lock operation

Always engage and disengage wheel locks on both sides simultaneously.

1. To Slow Down or Come to a Temporary Stop – Pull up slowly on the loop locks with four fingers until unit slows down or comes to a temporary stop.
2. To Temporarily Lock Wheels in Position – Press down on both loop locks with palms until you hear a “click” that will tell you wheels are locked.
3. To Release Wheel Locks – Push up on both loop locks with palms of hand to original position.

footrests

Installing the footrests

Align the two holes of the footrest on top of the frame pins and rotate forward until you hear the lock of the release lever.

To remove the footrest, push the release lever and swing the footrests to the side and lift the footrests off the frame pins.

Adjusting the footrests length

When adjusting the length of footrests the user should be sitting in rollator facing forward with feet on the foot plates. Push the pins on the footrest to lengthen. Raise or lower foot plate to desired position that will provide maximum support of thighs. Front rigging should not be so low that it causes uncomfortable pressure to the back of the thighs. Footrest should not be lower than 2-1/2" from the ground to ensure clearance.

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

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© 2006 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.