



VR110T

Width Adjustable Bath Board with Turntable



Fixing and Maintenance Instructions

This file is available to view and download as a PDF at www.aidapt.co.uk. Sight impaired customers can use a free PDF Reader (such as adobe.com/reader) to zoom in and increase the text size for improved readability.

INTRODUCTION

Thank you for purchasing the Width Adjustable Bath Board from Aidapt.

Please examine your product for any visible damage before using/installing. If you see any damage or suspect a fault, please do not use, but contact your supplier.

For maximum strength and safety, this plastic bath board is made from a single piece mould and is supplied with a handle. The contoured design provides added comfort and the well-spaced drainage holes ensure the water drains away quickly and efficiently. The Easy Self-Locating Fixing System makes adjustment, positioning and removal simple and safe.

If desired, you may purchase a green handle.

INTENDED USE

The Adjustable Bath Board is an aid to help accommodate users whilst bathing.

N.B. This equipment must be installed by a competent person and a risk assessment may be needed as to the suitability of the stool for the particular user.


Weight Limit: 180 kg (28 st.)

NB. Do not exceed the weight limit stated – doing so could put the user at risk.

FITTING YOUR BATH BOARD

Before attempting to fit your bath board please check that you have the following parts:

NB. If any parts are missing or damaged please contact your supplier.

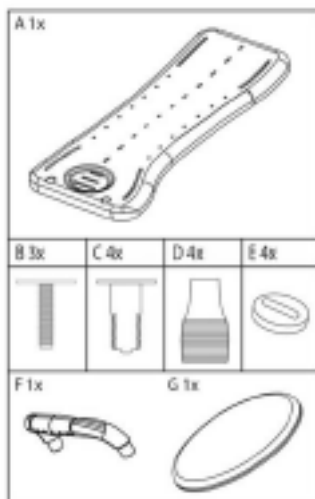

- A 1x Bathboard
- B 3x Plastic Bolt (Circular)
- C 4x Plastic Bolt (Rectangular)
- D 4x Bath board feet
- E 4x Plastic Washer
- F 1x Transfer Handle
- G 1x Swivel Seat


ATTACHING THE HANDLE

1. Insert the two ends of the blue handle into the circular holes at one end of the bath board. Insert the handle, which is slightly curved to one side, so that the top of the handle leans in, towards the middle of the bath board.
2. Using the plastic handle fixing screws supplied, insert each in turn into the corresponding hole on the bottom of the bath board and, holding the handle on the top of the bath board, turn the plastic fixing screws until they are finger tight only. NB. Do not use a screwdriver as this may damage the screws, if required a large coin may be used carefully to assist with turning the screws.




ATTACHING AND ADJUSTING THE FEET

1. Turn the bath board over so that the bottom of the board is facing upwards.
2. Insert each foot into the slot on the underside of the bath seat.
3. Turn the bath board over so that you can see the top of the board and locate the four corresponding slots for each foot.
4. Insert a plastic fixing screw through the slots into the narrow opening at the top of each foot and, holding the foot to prevent it from turning, turn the fixing screw until it is finger tight only. NB. Do not use a screwdriver as this may damage the screws, if required a large coin may be used carefully to assist with turning the screws.
5. Repeat for all four feet

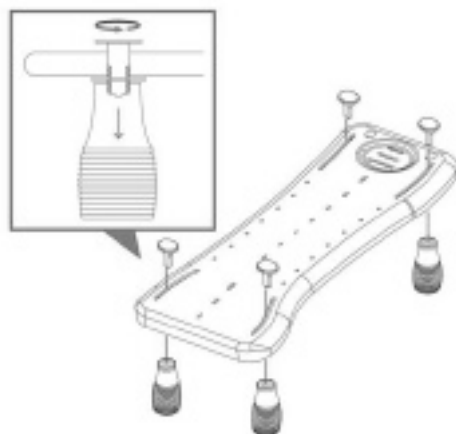

ATTACHING AND ADJUSTING THE FEET (CONTINUED)

6. Place the blue swivel disc into the hole in the middle of the bathboard. Whilst holding in place, use the circular bolt to secure in place from the bottom. Ensure you tighten until it is secured in place, but can still rotate. Do not over tighten, as the disc will not swivel.

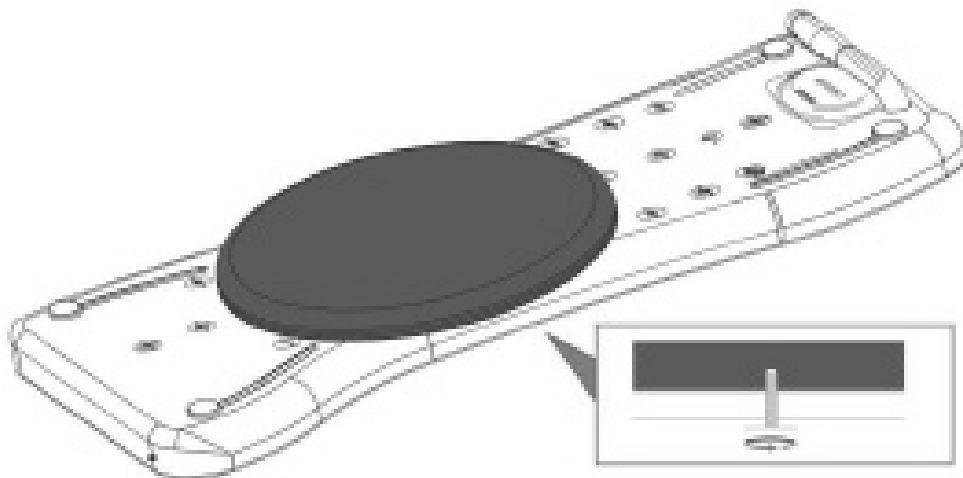

FITTING AND ADJUSTING THE BATH BOARD

1. Turn the board over with the feet facing up.
2. Loosen off the feet holding the plastic fixing screw in position.
3. Turn the board over so that the feet are facing down.
4. Place the board over the bath traversing its width.
5. Reach underneath the board and push both sets of feet outwards evenly until they are gripping the wall of the bath.
6. Tighten off the plastic fixing screws. N.B. It is only necessary to tighten the fixing screws finger tight.
7. Do not over tighten them as this can cause damage to the thread and cause the product to fail.
8. Double check the feet are friction tight to the wall of the bath before use.



Always check the tightness of the board before each use

Always repeat the installation every time the board is removed

CLEANING

The Bath Board should be cleaned using a non-abrasive cleaner or mild detergent with a soft cloth only. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may seriously damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment.

If to be disinfected by heat, one of the following three temperatures and duration of exposition can be used:

- a) a temperature of 90°C for 1 minute
- b) a temperature of 85°C for 3 minutes
- c) a temperature of 80°C for 10 minutes

MAINTENANCE

Please check the cradle periodically for any signs of damage and if in doubt contact your supplier.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety. This includes tightness of all nuts/bolts/castors and other screw in/bolt in/ push in components.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

TECHNICAL INFORMATION

Width	757mm
Depth.....	350mm
Height.....	95mm
Maximum User Weight Limit.....	28 stone (180kg)
Width Adjustment.....	463-680mm
Net Weight	2.38kg
Expected Lifetime	3 Years

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

A copy of the instructions leaflet can be downloaded from our website.